



CARE SURGICAL


CS ProneOne™ Head Support

*Suitable for patients 8 years+

The CS ProneOne™ offers support to your patient's forehead and cheeks. Ensure the eyebrows are visible to avoid direct pressure to the eyes.




Step 1 With patient supine, disconnect ET tubing

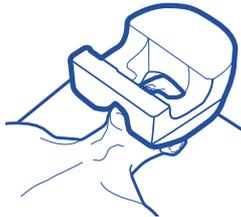


Step 2 Place cushion onto patient's face*



Step 3 Turn patient to prone position

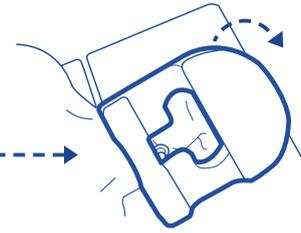

Step 4 Re-connect ET Tubing

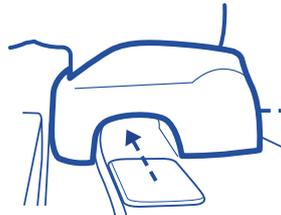

Step 5 Slide mirror into place



Step 6 Check face correctly positioned in cushion*



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, 69 Esker Woods Drive
Lucan, Co Dublin, K78PX45, Ireland



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120039




Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia

LOT Number:

Product Code: CSM-2520 Rev 11/06/21

The Forehead & Cheeks

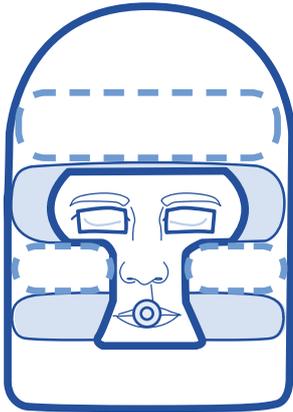

See the instructional video
at www.care-surgical.com

Possible problems

- + Pressure ulcers
- + Facial edema

Possible causes

- + Inadequate blood supply due to prolonged excessive pressure
- + Shear, affecting the blood flow to the forehead or cheeks
- + Friction injury due to a period of continuous movement
- + Patient incorrectly positioned in the face support
- + High risk patient (includes the elderly, diabetics, smokers and those taking certain medications, eg steroids)

Reduce the risk

- + Ensure the patient's face is supported on the bony part of their cheeks, not the soft tissue
- + Ensure the patient's eyebrows are clearly visible
- + The face support must not encroach into the periorbital area above or below the eye
- + Recheck the head support after turning the patient and at least every 30 minutes throughout the procedure
- + It may be necessary to lift and replace the patient's face during long procedures or if there has been movement



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, 69 Esker Woods Drive
Lucan, Co Dublin, K78PX45, Ireland



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120039

EC REP

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia

Product Code: CSM-2520 Rev 11/06/21

GTIN

- + 5060624120039

Description

- + Prone Face Cushion - Made from polyurethane foam with mirror strip

Intended use

- + Supports patients' faces in the prone position

Indications for Use

- + 2-Part product: See indications for use for CSM2520-1 and CSM2520-2
- + Instructions for use are packaged with the product

Warnings, Precautions, Contraindications & Adverse Effects

- + ProneOne Face Cushion (CSM2520-2):
 - + There is a 1/11,000 risk of permanent visual loss associated with prone positioning, irrespective of the head support used
 - + Incorrect positioning of the patient in this product can increase the risk of:
 - + POVL
 - + Pressure ulcers
 - + Neurological damage
 - + Venous pressure, which can lead to ischaemia
- + ProneOne Mirror Strip (CSM2520-1)
- + Breakage due to mishandling or poor storage could result in sharp edges causing injury to staff or patients

Packaging

- + Each 2-piece set is packaged in a recyclable polyethene bag
- + 6 sets are packaged into a cardboard box for sale

Transport & Storage

- + Transport and store in dry conditions and out of direct light where possible
- + Exposure to light may cause yellowing of areas of the product. This is purely cosmetic and will not affect the quality or the safety of the product