




# CARE SURGICAL

# CS Prone Face Mask

For use with Allen® C-Flex® Head positioning System.

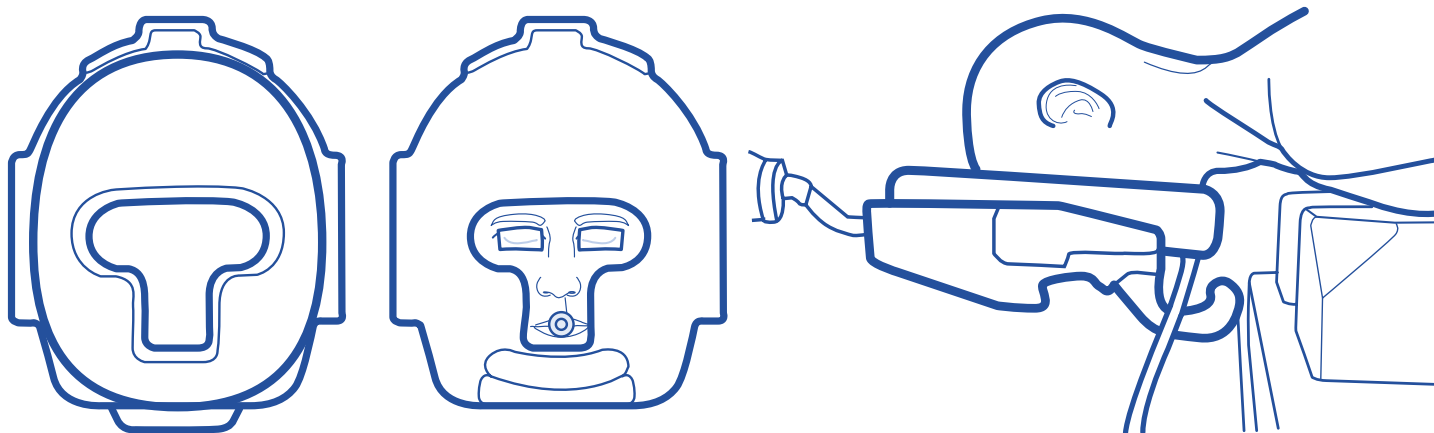

Follow the instructions supplied with the Allen® C-Flex® Head Positioning System to achieve a suitable neutral neckline

\*See overleaf for guidance on correct positioning



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
Marlborough  
MA 01752  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, 69 Esker Woods Drive  
Lucan, Co Dublin, K78PX45, Ireland



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120572




Australian Sponsor, Summit Surgical  
PO Box 60, Landsborough  
QLD 4550 Australia

LOT Number:

Product Code: CSM-2530 Rev 02 06/21

# The Forehead & Cheeks

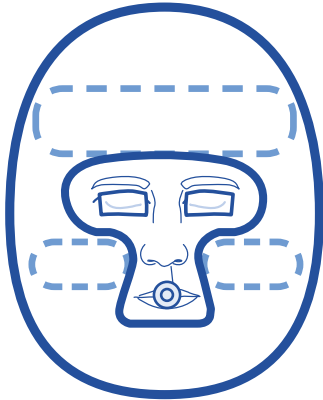

## Possible problems

- + Pressure ulcers
- + Facial edema

## Possible causes

- + Inadequate blood supply due to prolonged excessive pressure
- + Shear, affecting the blood flow to the forehead or cheeks
- + Friction injury due to a period of continuous movement
- + Patient incorrectly positioned in the face support
- + High risk patient (includes the elderly, diabetics, smokers and those taking certain medications, eg steroids)

## Reduce the risk

- + Ensure the patient's face is supported on the bony part of their cheeks, not the soft tissue
- + Ensure the patient's eyebrows are clearly visible
- + The face support must not encroach into the periorbital area above or below the eye
- + Recheck the head support after turning the patient and at least every 30 minutes throughout the procedure
- + It may be necessary to lift and replace the patient's face during long procedures or if there has been movement



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
Marlborough  
MA 01752  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, 69 Esker Woods Drive  
Lucan, Co Dublin, K78PX45, Ireland



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120572

EC REP

Australian Sponsor, Summit Surgical  
PO Box 60, Landsborough  
QLD 4550 Australia

Product Code: CSM-2530 Rev 02 06/21

## GTIN

- + 5060624120060

## Description

- + Prone Face Mask made from a 50mm Alexon 90 foam

## Intended use

- + Supports patients' faces in the prone position

## Indications for Use

- + Single use
- + Compatible with the Allen C-Prone Head Positioner
- + Distributes the pressure on the patient's cheeks, forehead and chin whilst in the prone position
- + Reduces the incidence of pressure ulcers and marking
- + Latex and phthalate free
- + Instructions for use are packaged with the product

## Warnings, Precautions, Contraindications & Adverse Effects

- + There is a 1/11,000 risk of permanent visual loss associated with prone positioning, irrespective of the head support used
- + Incorrect positioning of the patient in this product can increase the risk of:
  - + POVL
  - + Pressure ulcers
  - + Neurological damage
  - + Venous pressure, which can lead to ischaemia
- + Do not use if this product is torn or damaged

## Packaging

- + Each Unit is packaged in a recyclable polyethene bag
- + 5 units are packaged into a cardboard box for sale

## Transport & Storage

- + Transport and store in dry conditions and out of direct light where possible
- + Exposure to light may cause yellowing of areas of the product. This is purely cosmetic and will not affect the quality or the safety of the product