





CARE SURGICAL CS Adjustable Mirror Plus


Product code: CSM2567

For use with CS Prone Plus or CS Prone Child Face Cushions
to support patients' faces in the prone position


⚠ Only to be used by trained clinicians. Ensure product is secure before positioning patient.

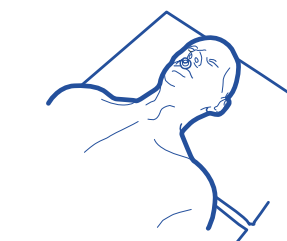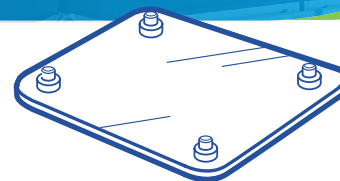


1 With patient supine,
disconnect ET tubing

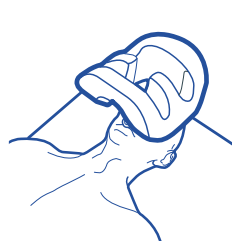


2 Place cushion onto
patient's face

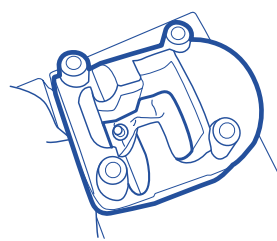


3 Place Helmet over
cushion

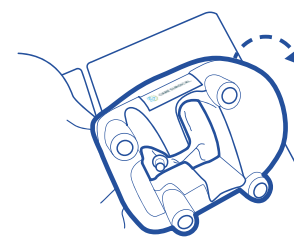

4 Turn patient to
prone position

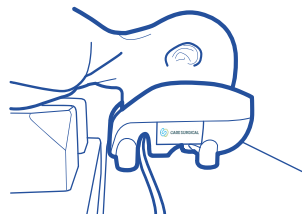


5 Re-connect ET Tubing

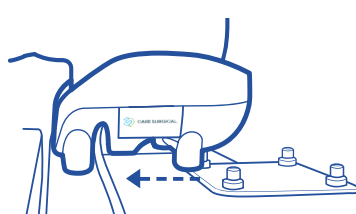

6 Lift helmet & locate
onto mirror's cogs






7 Check Face correctly
positioned in cushion

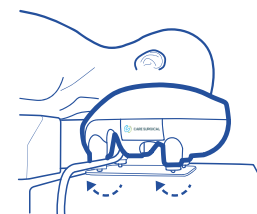

8 Adjust cogs to obtain
neutral neck line

Cleaning instructions:

Clean with warm soapy
water or standard
hospital wipes

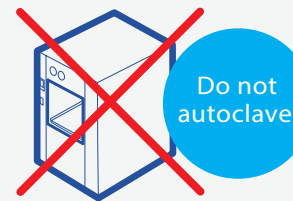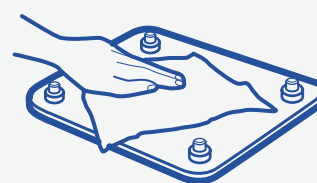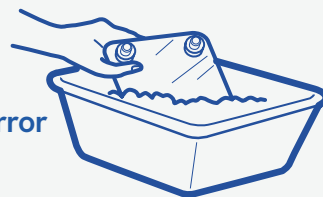

WARNING: Do not remove cogs from mirror



UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

⚠ **SAFE DISPOSAL:** customers should adhere to any local laws and regulations to the safe disposal of medical devices.
⚠ **DO NOT USE** products that appear to be damaged or worn.
In an event of product damage or patient injury, please contact Care Surgical.

GTIN
LOT



EC REP

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street
Lucan, Co Dublin, K78 X5P8 Ireland

Ref: CSM2567 - Product Label/IFU (Rev.07) 03/2025