




CARE SURGICAL

CS Contour Head Support Cradle

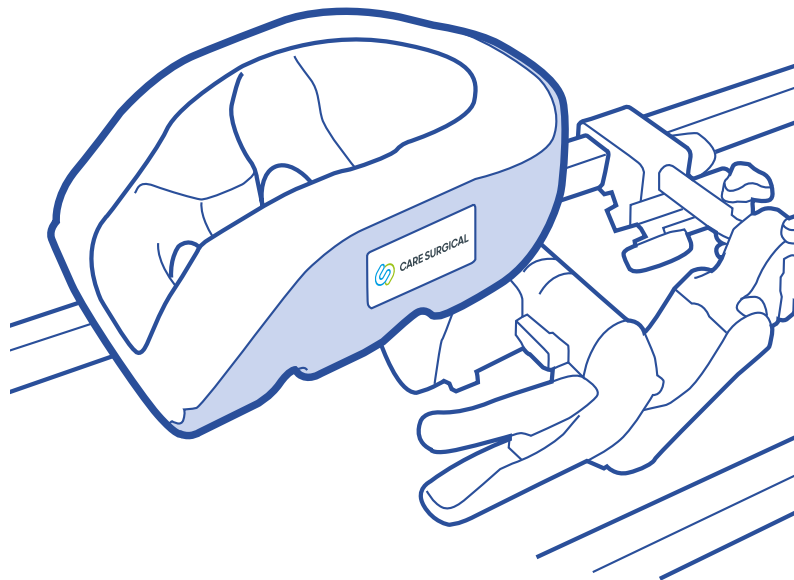

CS Contour Head Support Cradle

The CS Contour connects directly to the Allen® C-Flex® Head Positioning System.

Use only with the CS Prone Plus Face Cushion - Product Code CSM-2512 See

See C-Flex instructions for removing and attaching accessories



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, 69 Esker Woods Drive
Lucan, Co Dublin, K78PX45, Ireland



Do not reuse



CE Mark



Latex Free



Non Sterile




5 060624 120909

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia

LOT Number:

Product Code: CSM-2569 Rev 05 06/21

The Forehead & Cheeks

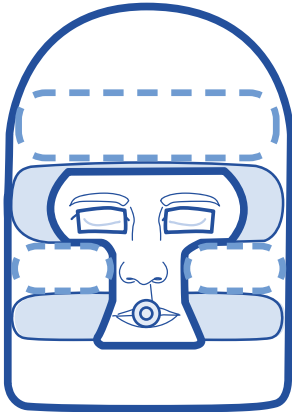

See the instructional video
at www.care-surgical.com

Possible problems

- + Pressure ulcers
- + Facial edema

Possible causes

- + Inadequate blood supply due to prolonged excessive pressure
- + Shear, affecting the blood flow to the forehead or cheeks
- + Friction injury due to a period of continuous movement
- + Patient incorrectly positioned in the face support
- + High risk patient (includes the elderly, diabetics, smokers and those taking certain medications, eg steroids)

Reduce the risk

- + Ensure the patient's face is supported on the bony part of their cheeks, not the soft tissue
- + Ensure the patient's eyebrows are clearly visible
- + The face support must not encroach into the periorbital area above or below the eye
- + Recheck the head support after turning the patient and at least every 30 minutes throughout the procedure
- + It may be necessary to lift and replace the patient's face during long procedures or if there has been movement



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, 69 Esker Woods Drive
Lucan, Co Dublin, K78PX45, Ireland



Do not reuse



CE Mark



Latex Free



Non Sterile

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia



5 060624 120909

EC REP

Product Code: CSM-2569 Rev 05 06/21

GTIN

- + 5060624120909

Description

- + Head support cradle made from ABS plastic

Intended use

- + Head support cradle for use in the prone position

Indications for Use

- + Reusable
- + Starburst style adapter to attach the cradle to the Allen C-Flex Head Positioning System
- + The cradle holds the CS Prone Plus Face Cushion or CS Child Face Cushion to maintain their integrity throughout prone procedures
- + Space for safe positioning of ET tubes
- + Clean the helmet with non-alcohol wipes or warm soapy water
- + Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)
- + Latex and phthalate free
- + Instructions for use are included in the box

Warnings, Precautions, Contraindications & Adverse Effects

- + Incorrect positioning of this product can increase the risk of:
 - + POVL
 - + Pressure ulcers
 - + Neurological damage
 - + Venous pressure, which can lead to ischaemia
- + This product should not be used if:
 - + There is a chip anywhere on the helmet with a sharp edge
 - + There is a crack in the helmet
 - + The shape of the helmet has become compromised/warped. This could cause lack of stability when placed on the mirror or table, or the Face Cushion not to sit squarely within the helmet
- + Do not immerse in water
- + Do not autoclave

Packaging

- + Each unit is packaged in a recyclable polyethene bag
- + 1 unit is packaged into a cardboard box for sale

Transport & Storage

- + Transport and store in dry conditions and out of direct light where possible
- + Do not store anything heavy on top