



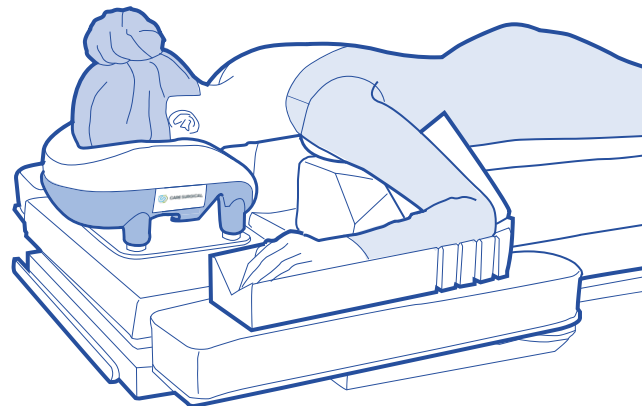

CARE SURGICAL

CS Arm Cushions (pair)

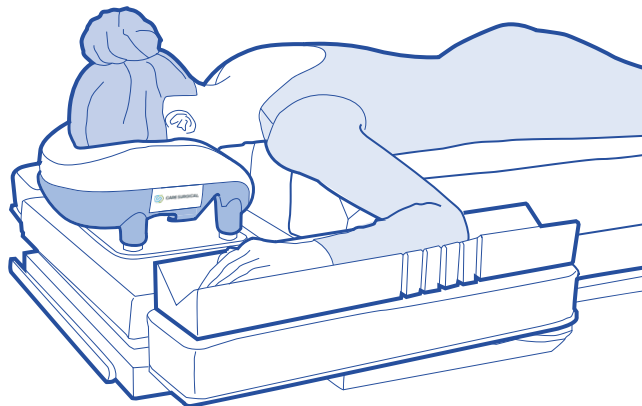

Arms forward and not extended beyond 90°, or arms secure by the patient's side.
No pressure on the axillary nerve. No body weight supported on the arms.



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, 69 Esker Woods Drive
Lucan, Co Dublin, K78PX45, Ireland



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120183


Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia

LOT Number:

Product Code: CSM-2575 Rev 08 06/21

EC REP

Compatible Products

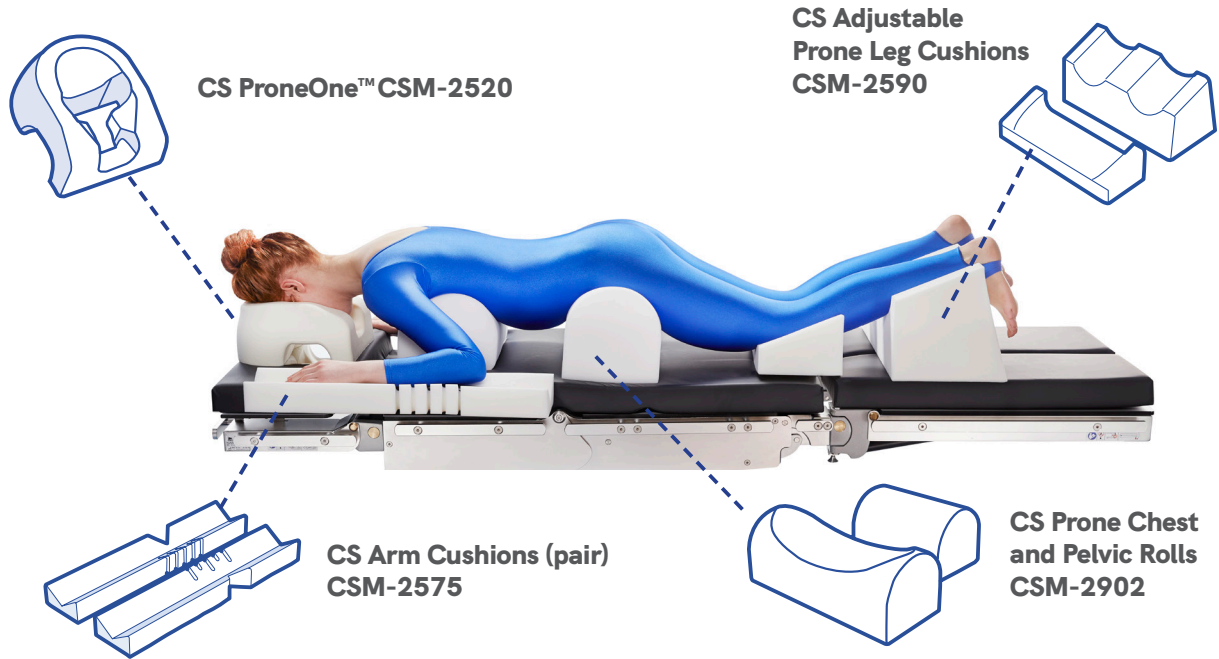

To see the full range of Care Surgical products, visit www.care-surgical.com



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, 69 Esker Woods Drive
Lucan, Co Dublin, K78PX45, Ireland



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120183

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia

Product Code: CSM-2575 Rev 08 06/21

EC REP

GTIN

- + 5060624120183

Description

- + Arm Supports - made from polyurethane foam

Intended use

- + Support patients' arms in the prone position

Indications for Use

- + Single use
- + Reduces and distributes pressure on the patient's arms while in the prone position
- + Reduces the risk of the patient experiencing marking or pressure ulcers
- + Notches allow manipulation of the arm support to support and protect the patient's upper arm and forearm, or just the forearm
- + Latex and phthalate free
- + Instructions for use are packaged with the product

Warnings, Precautions, Contraindications & Adverse Effects

- + Incorrect positioning of the patient in this product could result in the patient experiencing:
 - + Pressure ulcers
 - + Neurological damage, which can lead to temporary or permanent paralysis
 - + Venous pressure, which can lead to ischaemia
- + Do not use if this product is torn or damaged

Packaging

- + Each pair is packaged in a recyclable polyethene bag
- + 10 pairs are packaged into a cardboard box for sale

Transport & Storage

- + Transport and store in dry conditions and out of direct light where possible
- + Exposure to light may cause yellowing of areas of the product. This is purely cosmetic and will not affect the quality or the safety of the product