





CARE SURGICAL

CS Adjustable Prone Leg Cushions

Instructions for use

- + Ensure feet do not touch operating table
- + Ankles should extend beyond Shin support
- + Adjust position of the knee and Shin cushions according to the patient's stature
- + Knees should be touching the middle of the knee cushion

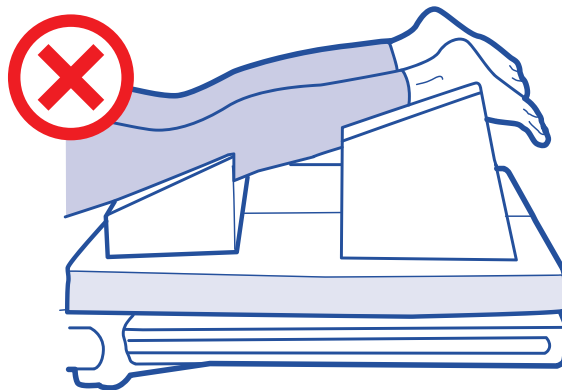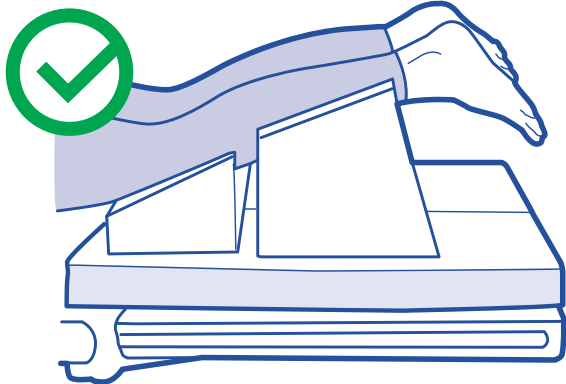

Manufacturer

UK: Care Surgical Ltd
 Unit 6 Ringtail Road
 Burscough, L40 8JY
 Tel: 01704 336671

USA: Care Surgical LLC
 Marlborough
 MA 01752
 Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
 Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity Assessed



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120213

LOT:

EC REP

Australian Sponsor, Summit Surgical
 PO Box 60, Landsborough
 QLD 4550 Australia

Product Code: CSM-2590 Rev.9 09/23

Head-to-Toe Check list

This check should be completed once the patient is in the prone position and before the surgical drapes are applied. At all stages, check that no tubing is trapped between the patient and the supports, and that all bony protrusions are padded.


The head

- + Eyes lightly taped, no padding or goggles
- + Face supported on the cheeks and forehead
- + No pressure on the nerves around the eyes
- + No pressure or minimal pressure on the chin



Chest, abdomen and pelvis

- + ECG electrodes positioned on the patient's back
- + Breasts positioned medially with minimum pressure
- + Adequate space to avoid compression of the abdomen
- + Male genitalia free of the pelvic support

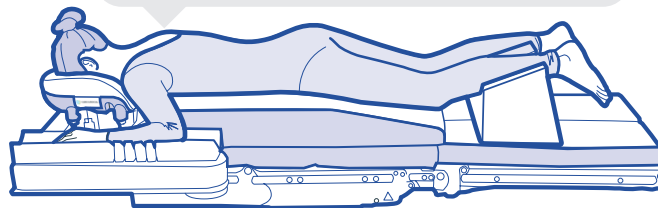

Neck

- + Chin and throat not touching the chest support
- + Natural neutral neckline

The arms

- + Arms forward and not abducted beyond 90°, or arms secure by the patient's side
- + No pressure on the axillary nerve
- + No body weight supported on the arms

To see the full range of Care Surgical Medical Products, visit www.care-surgical.com



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity
Assessed



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120213

EC REP

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia

ProductCode: CSM-2590 Rev.9 09/23