



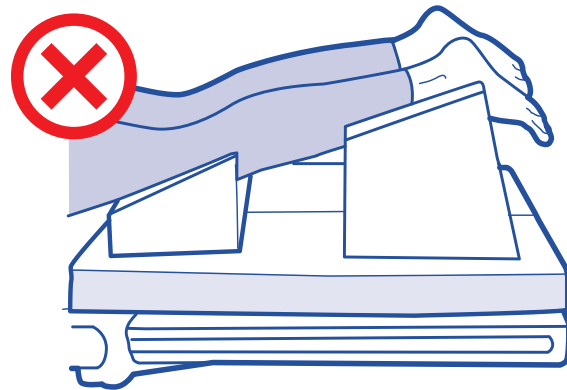

Instructions for use

- + Ensure feet do not touch operating table
- + Ankles should extend beyond Shin support
- + Adjust position of the knee and Shin cushions according to the patient's stature
- + Knees should be touching the middle of the knee cushion

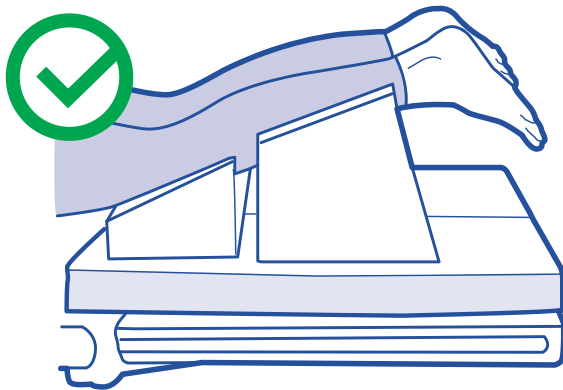

Manufacturer

UK: Care Surgical Ltd
 Unit 6 Ringtail Road
 Burscough, L40 8JY
 Tel: 01704 336671

USA: Care Surgical LLC
 Marlborough
 MA 01752
 Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, 69 Esker Woods Drive
 Lucan, Co Dublin, K78PX45, Ireland



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120213

Australian Sponsor, Summit Surgical
 PO Box 60, Landsborough
 QLD 4550 Australia

LOT Number:

Product Code: CSM-2590 Rev 08 06/21

Head-to-Toe Check list

This check should be completed once the patient is in the prone position and before the surgical drapes are applied. At all stages, check that no tubing is trapped between the patient and the supports, and that all bony protrusions are padded.


The head

- + Eyes lightly taped, no padding or goggles
- + Face supported on the cheeks and forehead
- + No pressure on the nerves around the eyes
- + No pressure or minimal pressure on the chin



Chest, abdomen and pelvis

- + ECG electrodes positioned on the patient's back
- + Breasts positioned medially with minimum pressure
- + Adequate space to avoid compression of the abdomen
- + Male genitalia free of the pelvic support

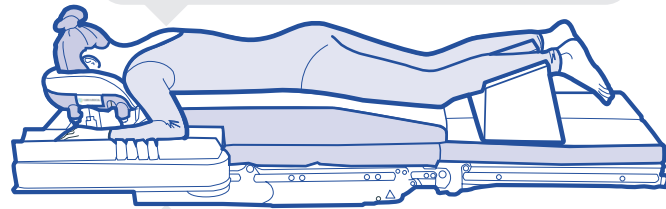

Neck

- + Chin and throat not touching the chest support
- + Natural neutral neckline

The arms

- + Arms forward and not abducted beyond 90°, or arms secure by the patient's side
- + No pressure on the axillary nerve
- + No body weight supported on the arms

To see the full range of Care Surgical Medical Products, visit www.care-surgical.com



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, 69 Esker Woods Drive
Lucan, Co Dublin, K78PX45, Ireland



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120213

EC REP

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia

LOT Number:

Product Code: CSM-2590 Rev 08 06/21

GTIN

- + 5060624120213

Description

- + Leg support - Made from polyurethane foam

Intended use

- + Supports patients' legs in the prone position

Indications for Use

- + Single Use
- + 2 part product
- + Part 1 supports the patient's knees (CSM 2590-1)
- + Part 2 supports the patient's shins and allows their ankles/feet to find their own position (CSM 2590-2)
- + Reduces the risk of pressure ulcers
- + The 2 parts can be moved together or apart to suit the stature of the patient
- + Latex and phthalate free
- + Instructions for use are packaged with the product

Warnings, Precautions, Contraindications & Adverse Effects

- + Incorrect positioning of the patient in this product could result in the patient experiencing:
 - + Pressure ulcers
 - + Neurological damage, which can lead to temporary or permanent paralysis
 - + Venous pressure, which can lead to ischaemia
- + Do not use if this product is torn or damaged

Packaging

- + Each Unit is packaged in a recyclable polyethene bag
- + 2 sets are packaged into a cardboard box for sale

Transport & Storage

- + Transport and store in dry conditions and out of direct light where possible
- + Exposure to light may cause yellowing of areas of the product. This is purely cosmetic and will not affect the quality or the safety of the product