




CARE SURGICAL

# CS Hip Supports (pair)

Patient Supports for use on the CS Spine Frame or spinal operating tables

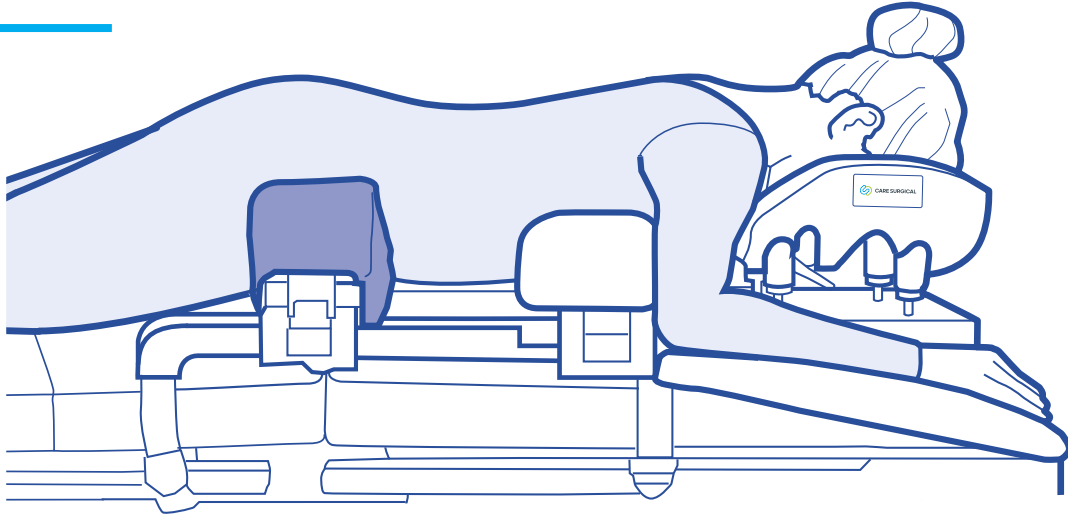

Refer to the label on the support to ensure it is placed in the correct position on the table.




Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
Marlborough  
MA 01752  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, 69 Esker Woods Drive  
Lucan, Co Dublin, K78PX45, Ireland

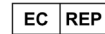

Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark




Maximum Weight

Australian Sponsor, Summit Surgical  
PO Box 60, Landsborough  
QLD 4550 Australia


LOT Number:




Product Code: CSM-2717 Rev 06 06/21

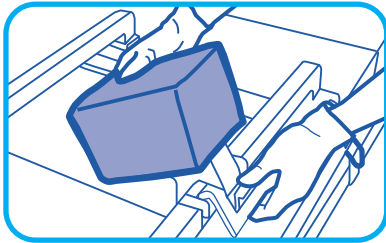

**1** Place the supports on the frame so that the brackets are over the bars

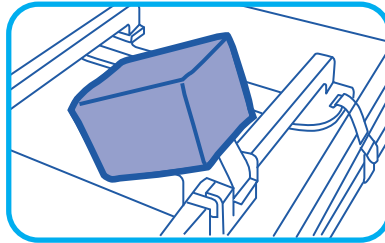

**2** Snap the locking clip shut to secure the support in position

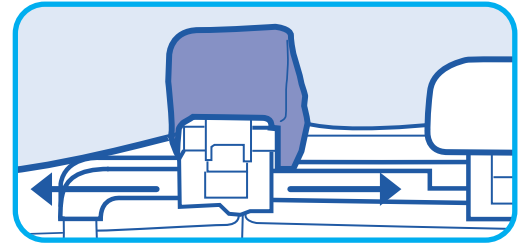

**3** To adjust the position of the supports, release the clip(s), raise the patient's chest or hips, slide the supports to the correct position, lock the clip(s)

Ensure the iliac crest is in the centre of the hip pads.

These products will support a patient weight of up to 250kg. Do not exceed the weight capacity of the operating table.


### Cleaning instructions:



Clean using standard hospital wipes



Do not immerse in water

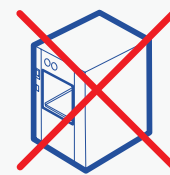

Do not autoclave



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
Marlborough  
MA 01752  
Tel: 866-243-4107



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark



Maximum Weight

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, 69 Esker Woods Drive  
Lucan, Co Dublin, K78PX45, Ireland



Australian Sponsor, Summit Surgical  
PO Box 60, Landsborough  
QLD 4550 Australia



5 060624 120626

Product Code: CSM-2717 Rev 06 06/21

## GTIN

- + 5060624120626

## Description

- + See CSM2710

## Intended use

- + See CSM2710

## Indications for Use

- + See CSM2710

## Warnings, Precautions, Contraindications & Adverse Effects

- + See CSM2710

## Packaging

- + Each Unit is packaged in a recyclable polyethene bag
- + 1 pair is packaged into a cardboard box for sale

## Transport & Storage

- + Transport and store in dry conditions and out of direct light where possible
- + Avoid storing where sharp objects could pierce the skin of the product