




CARE SURGICAL

CS Heel Supports (Pair)

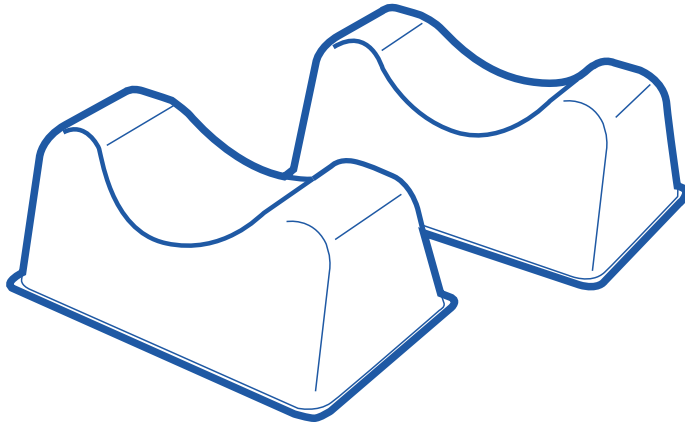

CS Heel Supports (Pair)

Distributes pressure on the back of the patients ankles in the supine position.
Enhances infection control.



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, 69 Esker Woods Drive
Lucan, Co Dublin, K78PX45, Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark



5 060624 120695

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia

LOT Number:

Product Code: CSM-2807 Rev 05 06/21




CARE SURGICAL


CS Heel Supports (Pair)

Instructions for use

- + Place the ankles in the centre of the support for minimum pressure in the supine position

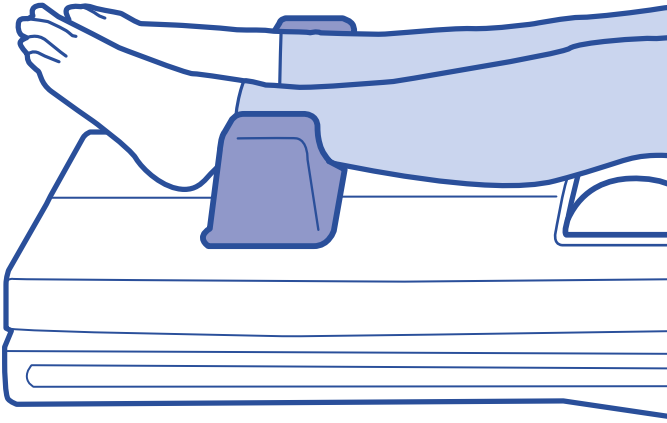

Cleaning instructions:

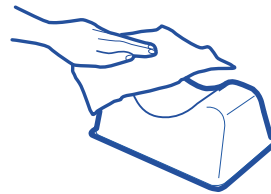

Clean using standard hospital wipes



Do not immerse in water



Do not autoclave




Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, 69 Esker Woods Drive
Lucan, Co Dublin, K78PX45, Ireland

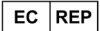

Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia



5 060624 120695

Product Code: CSM-2807 Rev 05 06/21

GTIN

- + 5060624120695

Description

- + Contoured blocks made from polyurethane foam with a wipeable skin covering

Intended use

- + To support the patient's ankles in the supine position

Indications for Use

- + Reusable
- + Distributes pressure on the back of the patient ankles in the supine position
- + Reduces the incidence of pressure related injury to the heels
- + Clean using non-alcohol hospital wipes
- + Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)
- + Instructions for use are packaged with the product

Warnings, Precautions, Contraindications & Adverse Effects

- + Incorrect positioning of the patient on this product can cause:
 - + Pressure marking or ulcers
 - + Neurological damage, which can cause paraplegia
 - + Venous pressure, which can lead to ischaemia
- + Examine the pad for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced
- + Do not immerse in water
- + Do not autoclave

Packaging

- + Each Unit is packaged in a recyclable polyethene bag
- + 1 pair is packaged into a cardboard box for sale

Transport & Storage

- + Transport and store in dry conditions and out of direct light where possible
- + Avoid storing where sharp objects could pierce the skin of the product