




CARE SURGICAL

CS Knee Rings (Pair)

Instructions for use

- + Place the knees in the centre of the rings for minimum pressure

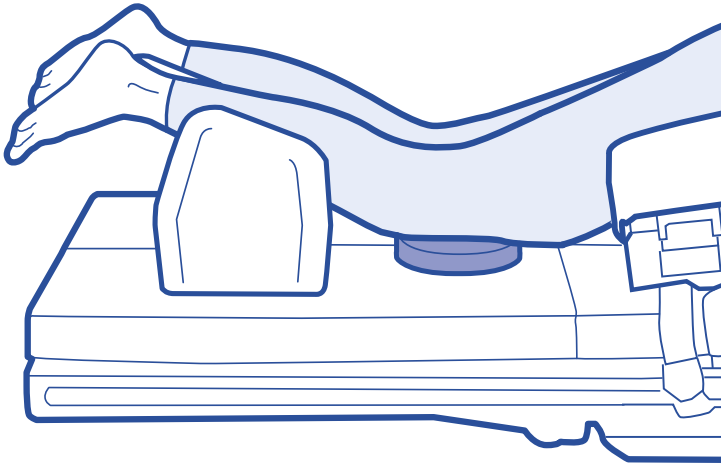

Cleaning instructions:



Clean using standard hospital wipes



Do not immerse in water



Do not autoclave



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, 69 Esker Woods Drive
Lucan, Co Dublin, K78PX45, Ireland



Medical Device



Reusable



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark



5 060624 121128

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia

LOT Number:

Product Code: CSM-2632 Rev 06 08/21

Head-to-Toe Check list

This check should be completed once the patient is in the prone position and before the surgical drapes are applied. At all stages, check that no tubing is trapped between the patient and the supports, and that all bony protrusions are padded.

The head

- + Eyes lightly taped, no padding or goggles
- + Face supported on the cheeks and forehead
- + No pressure on the nerves around the eyes
- + No pressure or minimal pressure on the chin




Neck

- + Chin and throat not touching the chest support
- + Natural neutral neckline

Chest, abdomen and pelvis

- + ECG electrodes positioned on the patient's back
- + Breasts positioned medially with minimum pressure
- + Adequate space to avoid compression of the abdomen
- + Male genitalia free of the pelvic support

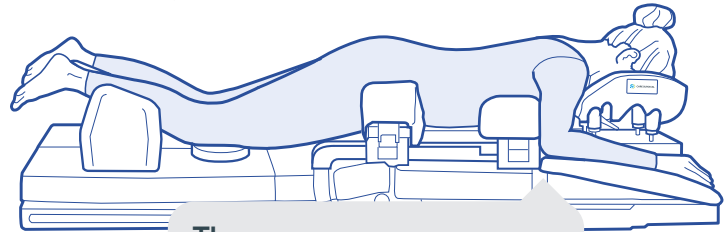

The arms

- + Arms forward and not abducted beyond 90°, or arms secure by the patient's side
- + No pressure on the axillary nerve
- + No body weight supported on the arms

To see the full range of Care Surgical Medical Products, visit www.care-surgical.com



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, 69 Esker Woods Drive
Lucan, Co Dublin, K78PX45, Ireland



Medical Device



Reusable



Latex Free



Non Sharps



UK Conformity Assessed



CE Mark

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia



Product Code: CSM-2632 Rev 06 08/21



GTIN

- + 5060624121128

Description

- + Shin supports made from polyurethane foam with a wipeable skin covering

Intended use

- + Supports patients' lower legs in the prone position

Indications for Use

- + Support the patient's shins in the prone position and allows their ankles/feet to find their own position
- + Reduce the risk of pressure ulcers
- + Latex and phthalate free
- + Reusable: Clean using non-alcohol hospital wipes
- + Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)
- + Instructions for use are packaged with the product

Warnings, Precautions, Contraindications & Adverse Effects

- + Incorrect positioning of the patient on this product could result in the patient experiencing:
 - + Pressure ulcers
 - + Neurological damage, which can lead to temporary or permanent paralysis
 - + Venous pressure, which can lead to ischaemia
- + Examine the product for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced
- + Do not immerse in water
- + Do not autoclave

Packaging

- + Each Unit is packaged in a recyclable polyethene bag
- + 1 pair is packaged into a cardboard box for sale

Transport & Storage

- + Transport and store in dry conditions and out of direct light where possible
- + Avoid storing where sharp objects could pierce the skin of the product