




# CARE SURGICAL

# CS Prone Chest & Pelvic Rolls

Reduce pressure marks by using the Prone Chest & Pelvic Rolls.

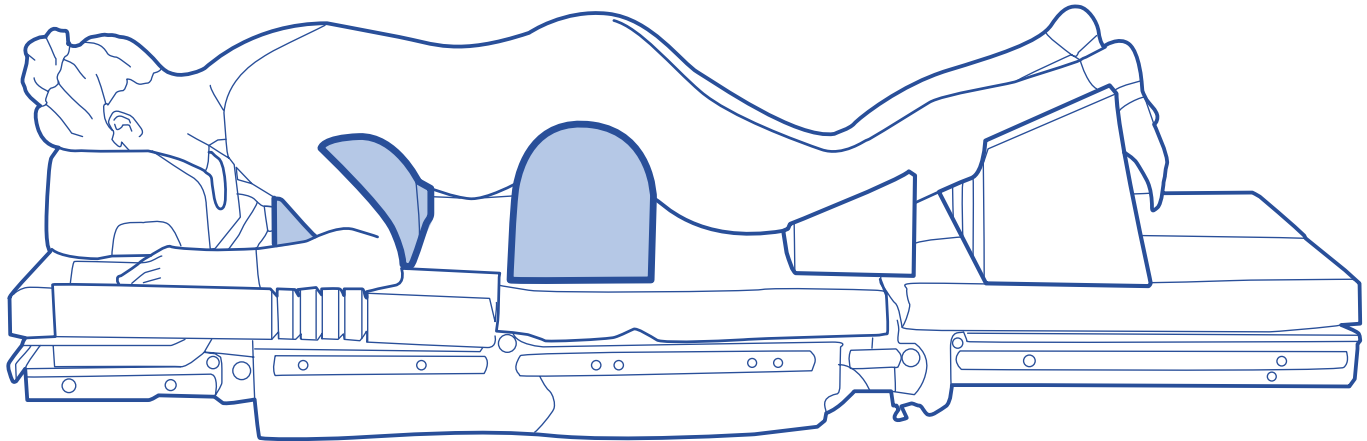

Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
Marlborough  
MA 01752  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, 69 Esker Woods Drive  
Lucan, Co Dublin, K78PX45, Ireland



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120442

Australian Sponsor, Summit Surgical  
PO Box 60, Landsborough  
QLD 4550 Australia

LOT Number:

Product Code: CSM-2902 Rev 04 06/21

## GTIN

+ 5060624120442

## Description

+ Dome shapes supports made from polyurethane foam

## Intended use

- + 2-part product: CS Chest Roll (CSM 2900)  
To support the patient's chest in the prone position
- + CS Pelvic Roll (CSM 2901)  
To support the patient's pelvis in the prone position

## Indications for Use

CS Chest Roll (CSM 2900)

- + Single use - disposal after use with regular clinical waste
- + Raises the patient's chest in the prone position to assist
- + Venous return and diaphragmatic excursion return
- + Achieving a natural neutral neckline
- + For patients weighing up to 200kg
- + Latex and phthalate free

CS Pelvic Roll (CSM 2901)

- + Single use
- + Raises the patient's pelvis in the prone position to assist venous return and diaphragmatic excursion return and for achieving correct positioning for the legs
- + For patients weighing up to 200kg
- + Latex and phthalate free
- + Instructions for use are packaged with the product

## Warnings, Precautions, Contraindications & Adverse Effects

- + Incorrect positioning of the patient on this product can cause:
  - + Pressure marking or ulcers
  - + Neurological damage, which can cause paraplegia
  - + Venous pressure, which can lead to ischaemia
- + Do not use if this product is torn or damaged

## Packaging

- + Each Unit is packaged in a recyclable polyethene bag
- + 2 set are packaged into a cardboard box for sale

## Transport & Storage

- + Transport and store in dry conditions and out of direct light where possible
- + Exposure to light may cause yellowing of areas of the product. This is purely cosmetic and will not affect the quality or the safety of the product