CARE SURGICAL
Specialists in Patient Positioning

INSTRUCTIONS FOR USE

CSM2512

CS Prone Plus Face Cushion


CARE SURGICAL

CS Prone Plus Face Cushion

*Suitable for patients 8 years+
Product code: CSM2512

Ensure the eyebrows are visible to avoid direct pressure to the eyes.
Use with CSM2565 (CSM2567 & CSM2565-1) and other compatible head support systems


Only to be used by trained clinicians.

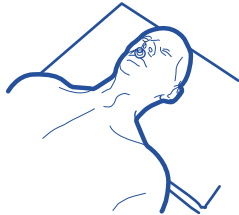
1 With patient supine, disconnect ET tubing

2 Place cushion onto patient's face*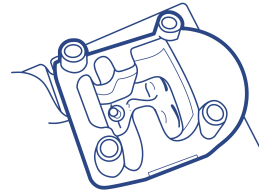


3 Place helmet (CSM2565-1) over cushion

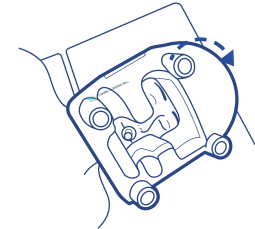
4 Turn patient to prone position


5 Re-connect ET Tubing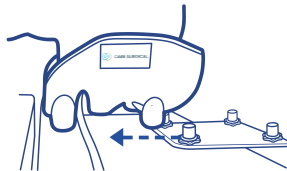

6 Lift helmet & locate onto mirror's cogs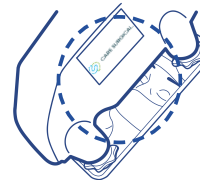

7 Check face is correctly positioned in cushion*

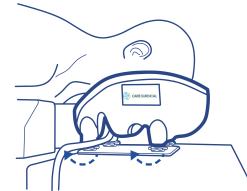8 Adjust cogs to obtain neutral neck line

UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

DO NOT REUSE! If reused injury to patient could occur due to issues with infection control and deterioration of the product.
SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.
DO NOT USE products that appear to be damaged or worn.
In an event of product damage or patient injury, please contact Care Surgical.

5 060624 120008

EU REP

The Forehead & Cheeks

Possible problems


- + Pressure ulcers
- + Facial edema

Possible causes

- + Inadequate blood supply due to prolonged excessive pressure
- + Shear, affecting the blood flow to the forehead or cheeks
- + Friction injury due to a period of continuous movement
- + Patient incorrectly positioned in the face support
- + High risk patient (includes the elderly, diabetics, smokers and those taking certain medications, eg steroids)

Reduce the risk

- + Ensure the patient's face is supported on the bony part of their cheeks, not the soft tissue
- + Ensure the patient's eyebrows are clearly visible
- + The face support must not encroach into the periorbital area above or below the eye
- + Recheck the head support after turning the patient and at least every 30 minutes throughout the procedure
- + It may be necessary to lift and replace the patient's face during long procedures or if there has been movement

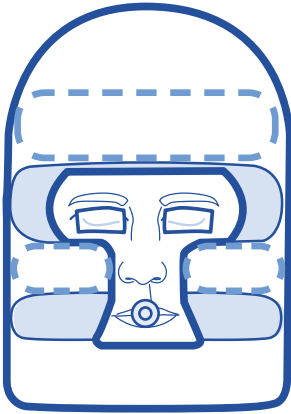See the instructional video
at www.care-surgical.com

UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

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