CARE SURGICAL
Specialists in Patient Positioning

INSTRUCTIONS FOR USE

CSM2536V

**CS Prone Face Block 7" - Left & Right Intubation
(Compression Packed)**

CARE SURGICAL


Product Code: CSM2536V

CS Prone Face Block 7" - Left & Right Intubation Slot (Compression packed*)


*A long storage life of compressed packed products can comprise the product's integrity (e.g height and/or the rate in which the product fully inflates.)

⚠ Only to be used by trained clinicians.


The Prone Block offers support to your patient's forehead and cheeks. Ensure the eyebrows are visible to avoid direct pressure to the eyes.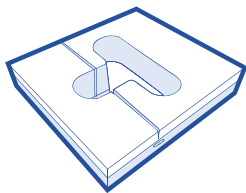


1 Remove from bag 1 hour before start of procedure**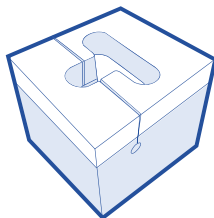


2 With patient supine, place cushion on their face***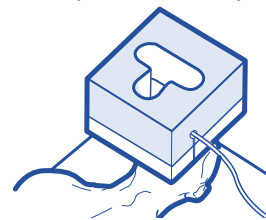

3 Feed the tubing up either slot & through the hole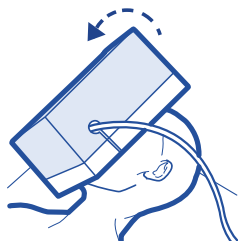

4 Turn patient to prone position

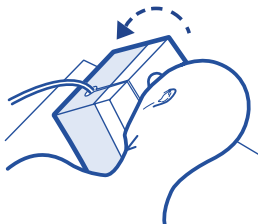5 Once turned Ensure neutral neck line

****Allow a minimum of 1 hour for full inflation. During this time, gently stretch the foam by hand periodically to support full expansion.**

***** See overleaf for guidance on correct positioning**

UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

EU REP

⚠ **DO NOT REUSE!** If reused injury to patient could occur due to issues with infection control and deterioration of the product.
⚠ **SAFE DISPOSAL:** customers should adhere to any local laws and regulations to the safe disposal of medical devices.
⚠ **DO NOT USE** products that appear to be damaged or worn.
In an event of product damage or patient injury, please contact Care Surgical.

5 060624 120732

The Forehead & Cheeks

Possible problems


- + Pressure ulcers
- + Facial edema

Possible causes

- + Inadequate blood supply due to prolonged excessive pressure
- + Shear, affecting the blood flow to the forehead or cheeks
- + Friction injury due to a period of continuous movement
- + Patient incorrectly positioned in the face support
- + High risk patient (includes the elderly, diabetics, smokers and those taking certain medications, eg steroids)

Reduce the risk

- + Ensure the patient's face is supported on the bony part of their cheeks, not the soft tissue
- + Ensure the patient's eyebrows are clearly visible
- + The face support must not encroach into the periorbital area above or below the eye
- + Recheck the head support after turning the patient and at least every 30 minutes throughout the procedure
- + It may be necessary to lift and replace the patient's face during long procedures or if there has been movement

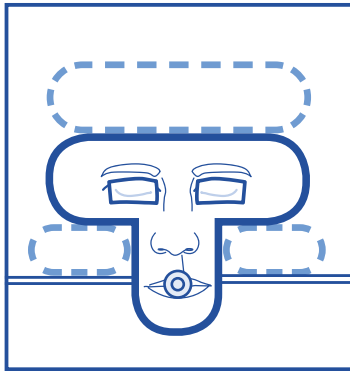UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

CARE SURGICAL

Artikelnummer: CSM2536V

CS Prone Face Block 7" - Left & Right Intubation Slot (Compression packed*)

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