CARE SURGICAL
Specialists in Patient Positioning

INSTRUCTIONS FOR USE

CSM2550V

**CS Prone Face Block 5" - Right Intubation Slot
(Compression Packed)**

CARE SURGICAL


Product Code: CSM2550V

CS Prone Face Block 5" - Right Intubation Slot (compression packed*)


*A long storage life of compressed packed products can compromise the product's integrity (e.g height and/or the rate in which the product fully inflates).

⚠️ Only to be used by trained clinicians.


The Prone Block offers support to your patient's forehead and cheeks Ensure the eyebrows are visible to avoid direct pressure to the eyes.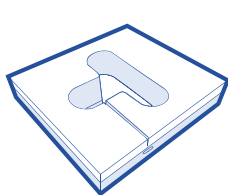


1 Remove from bag 1 hour before start of procedure**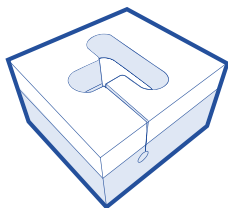


2 With patient supine, place cushion on their face**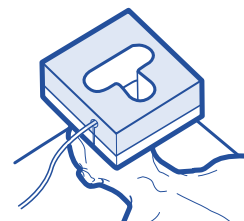

3 Feed the tubing up the slot & through the hole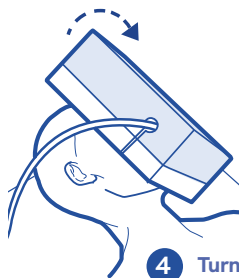

4 Turn patient to prone position

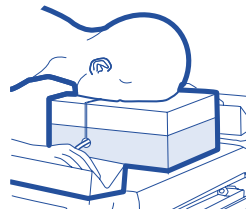5 Once turned Ensure neutral neck line

**Allow a minimum of 1 hour for full inflation. During this time, gently stretch the foam by hand periodically to support full expansion.

*** See overleaf for guidance on correct positioning

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

DO NOT REUSE! If reused injury to patient could occur due to issues with infection control and deterioration of the product.
SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.
DO NOT USE products that appear to be damaged or worn.
In an event of product damage or patient injury, please contact Care Surgical.

5 060624 120091

The Forehead & Cheeks

Possible problems


- + Pressure ulcers
- + Facial edema

Possible causes

- + Inadequate blood supply due to prolonged excessive pressure
- + Shear, affecting the blood flow to the forehead or cheeks
- + Friction injury due to a period of continuous movement
- + Patient incorrectly positioned in the face support
- + High risk patient (includes the elderly, diabetics, smokers and those taking certain medications, eg steroids)

Reduce the risk

- + Ensure the patient's face is supported on the bony part of their cheeks, not the soft tissue
- + Ensure the patient's eyebrows are clearly visible
- + The face support must not encroach into the periorbital area above or below the eye
- + Recheck the head support after turning the patient and at least every 30 minutes throughout the procedure
- + It may be necessary to lift and replace the patient's face during long procedures or if there has been movement

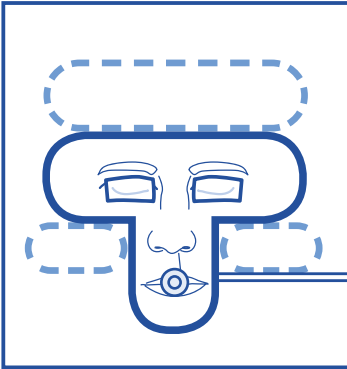UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

CARE SURGICAL

Artikelnummer: CSM2550V

CS Prone Face Block 5" - Right Intubation Slot (compression packed*)

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