




Technical Data Sheet

CSM2560 CS Prone Head Support System™

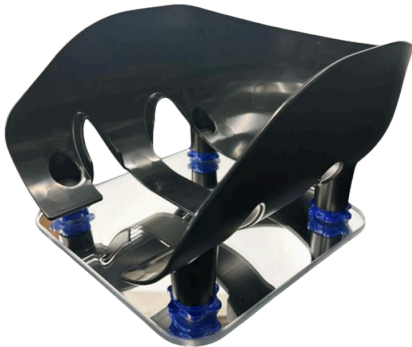

Description

- Reusable
- Helmet holds the CS Prone Face Cushion (CSM2510) to maintain their integrity throughout prone procedures
- Space for safe positioning of ET tubes
- The helmet can be used with or without the mirror
- The mirror enables the clinician to monitor the patient's eyes and tubing during the procedure
- The cogs fitted to the mirror are used to adjust the height or angle of the helmet to achieve the safest natural neutral neckline

Application Area

- OR
- Anaesthesia
- ICU

Measurements

Helmet:

- Length: 317mm/12.48"
- Width: 233mm/9.17"
- Height: 171mm/6.73"

Mirror:

- Length: 220mm/8.6"
- Width: 220 mm/8.6"
- Height: 4mm/0.16"

Material Composition

- Helmet made from ABS Plastic
- Acrylic mirror, which incorporates 4 adjustable cogs
- Aluminium 6061

Order Information

Product Code	Product Name	Qty
CSM2560	CS Prone Head Support System™	1 System

Cleaning

- Clean with standard hospital wipes

Packaging Information

- Each set is packaged in a recyclable polyethylene bag
- Each set is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top

Contraindications

Incorrect positioning of the patient on this product can cause:

- POVL
 - pressure ulcers
 - neurological damage
 - venous pressure, which can lead to ischaemia
-
- Do not use if there is any damage on the helmet or mirror
 - Do not autoclave
 - Only to be used by trained medical personnel

Further Information

- Disposable/Reusable: Reusable
- Not fully radiolucent- cog screws may be visible on x-rays
- Sterility: N
- CE/FDA/TGA/UKCA:Y
- MD Class: 1
- UDI-DI/GTIN 5060624120114

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Ref: CSM2560- TDS (Rev.05) 06/2026