CARE SURGICAL
Specialists in Patient Positioning

INSTRUCTIONS FOR USE

CSM2561

CS Prone Smart Head Support System (Helmet + Mirror)

CARE SURGICAL

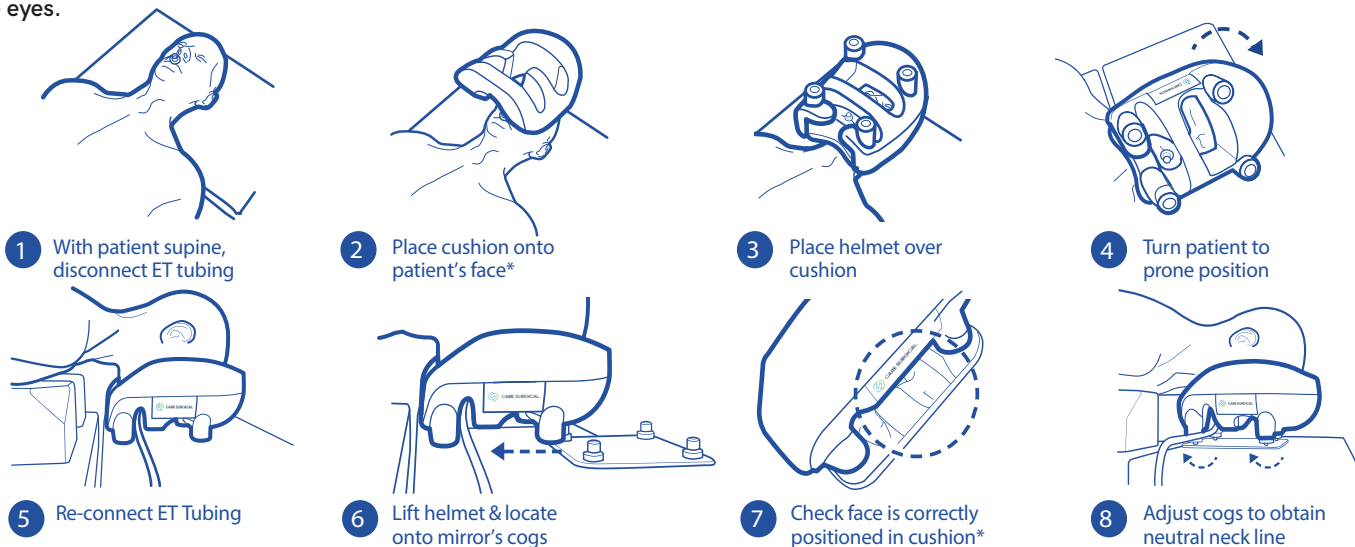
Product Code: CSM2561

CS Prone Smart Head Support System (Helmet & Mirror)

*Suitable for patients 8 years+. Only to be used with the CS Absolute Prone Face Cushions

⚠ Only to be used by trained clinicians. Ensure product is secure before positioning patient.

The CS Prone Smart Head Support System (Helmet and Mirror) used with compatible face cushion offers support to your patient's forehead and cheeks whilst ensuring a natural neckline and assisting with patient monitoring. Ensure the eyebrows are visible to avoid direct pressure to the eyes.

Warning: Do not remove cogs from mirror Ensure product is secure before positioning patient.

UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

EU REP

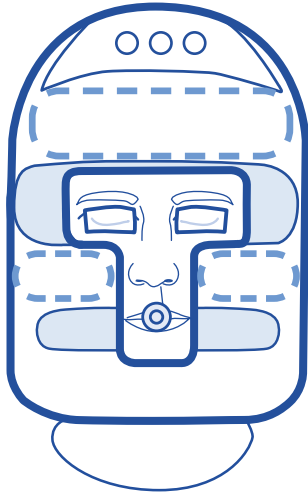
SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.
DO NOT USE products that appear to be damaged or worn.
In an event of product damage or patient injury, please contact Care Surgical.

5 060624 123207

The Forehead & Cheeks

See the instructional video
at www.care-surgical.com

Possible problems

- + Pressure ulcers
- + Facial edema

Possible causes

- + Inadequate blood supply due to prolonged excessive pressure
- + Shear, affecting the blood flow to the forehead or cheeks
- + Friction injury due to a period of continuous movement
- + Patient incorrectly positioned in the face support
- + High risk patient (includes the elderly, diabetics, smokers and those taking certain medications, eg steroids)

Reduce the risk

- + Ensure the patient's face is supported on the bony part of their cheeks, not the soft tissue
- + Ensure the patient's eyebrows are clearly visible
- + The face support must not encroach into the periorbital area above or below the eye
- + Recheck the head support after turning the patient and at least every 30 minutes throughout the procedure
- + It may be necessary to lift and replace the patient's face during long procedures or if there has been movement

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main
Street, Lucan Co. Dublin, K78 X5P8 Ireland

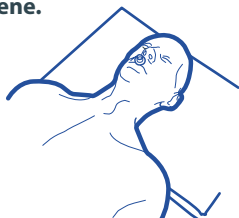
CARE SURGICAL

Produktkode: CSM2561

CS Prone Smart Head Support System (Helmet & Mirror)

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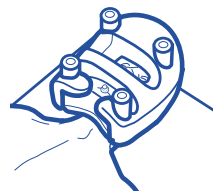
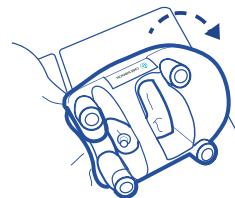
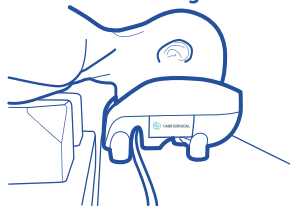
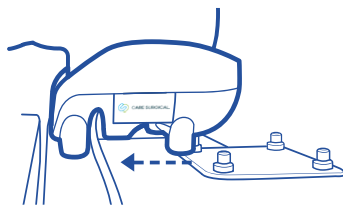
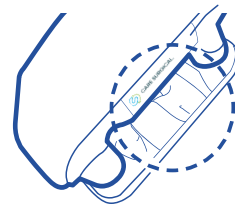
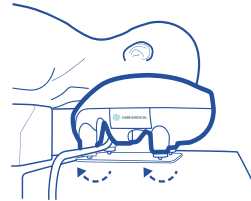
www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 XSP8 Ireland

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