



Technical Data Sheet

CSM2565 CS Prone Plus Head Support System™



Description

- Reusable
- Helmet holds the CS Prone Plus Face Cushion (CSM2512) or CS Small Face Cushion (CSM2514) to maintain their integrity throughout prone procedures
- Space for safe positioning of ET tubes.
- The helmet can be used with or without the mirror
- The mirror enables the clinician to monitor the patient's eyes and tubing during the procedure
- The cogs fitted to the mirror are used to adjust the height or angle of the helmet to achieve the safest natural neutral neckline

Application Area

- OR
- Anaesthesia
- ICU

Measurements

Helmet:

- Length: 317mm/12.48"
- Width: 233mm/9.17"
- Height: 155mm/ 6.10"

Mirror:

- Length: 220 mm/ 8.6"
- Width: 220 mm/8.6"

Material Composition

- Helmet made from ABS Plastic
- Acrylic mirror, which incorporates 4 adjustable cogs
- Aluminium 6061

Cleaning

- Clean with standard hospital wipes

Order Information

| Product Code | Product Name | Qty |
|--------------|------------------------------------|----------|
| CSM2565 | CS Prone Plus Head Support System™ | 1 System |

Packaging Information

- Each set is packaged in a recyclable polyethylene bag
- Each set is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top

Contraindications

Incorrect positioning of the patient on this product can cause:

- POVL
- pressure ulcers
- neurological damage
- venous pressure, which can lead to ischaemia

- Do not use if there is any damage on the helmet or mirror
- Do not autoclave
- Only to be used by trained medical personnel

Further Information

- Disposable/Reusable: Reusable
- Not fully radiolucent- cog screws may be visible on x-rays
- Sterility: N
- CE/FDA/UKCA:Y
- MD Class: 1
- UDI-DI/GTIN No: 5060624120121

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Australian Sponsor:

MDSS AU Pty Ltd, Ground Floor, Suite 55,
97-99 Bathurst Street, Sydney, NSW 2000, Australia



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark