**CARE SURGICAL**  
Specialists in Patient Positioning

## **INSTRUCTIONS FOR USE**

**CSM2565**

**CS Prone Plus Head Support System (Helmet & Mirror)**

# CARE SURGICAL

Product Code: CSM2565

# CS Prone Plus Head Support System (Helmet & Mirror)

\*Suitable for patient's 8 years+. To only be used with the CS Prone Face Cushion CSM2512

⚠ Only to be used by trained clinicians. Ensure product is secure before positioning patient.

The CS Prone Plus Head Support System used with The CS Prone Plus Face Cushion offers support to your patient's forehead and cheeks whilst ensuring a neutral neckline and assisting with patient monitoring. Ensure the eyebrows are visible to avoid direct pressure to the eyes.

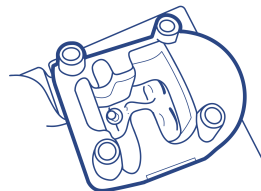
**Warning: Do not remove cogs from mirror**

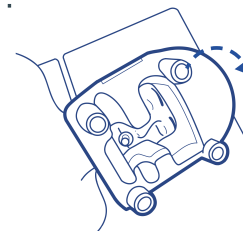
**1** With patient supine, disconnect ET tubing

**2** Place cushion onto patient's face\*

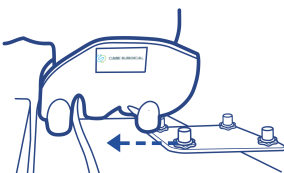
**3** Place helmet over cushion

**4** Turn patient to prone position

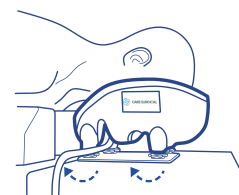
**5** Re-connect ET Tubing

**6** Lift helmet & locate onto mirror's cogs

**7** Check face is correctly positioned in cushion\*

**8** Adjust cogs to obtain neutral neck line

\*See overleaf for guidance on correct positioning

UK: Care Surgical Ltd  
Unit 6 Ringtail Road,  
Burscough,  
L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

EU REP

⚠ **SAFE DISPOSAL:** customers should adhere to any local laws and regulations to the safe disposal of medical devices.  
⚠ **DO NOT USE** products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.

5 060624 120121

# The Forehead & Cheeks

## Possible problems

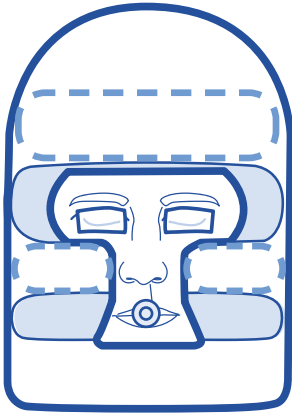
- + Pressure ulcers
- + Facial edema

## Possible causes

- + Inadequate blood supply due to prolonged excessive pressure
- + Shear, affecting the blood flow to the forehead or cheeks
- + Friction injury due to a period of continuous movement
- + Patient incorrectly positioned in the face support
- + High risk patient (includes the elderly, diabetics, smokers and those taking certain medications, eg steroids)

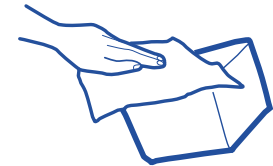
## Reduce the risk

- + Ensure the patient's face is supported on the bony part of their cheeks, not the soft tissue
- + Ensure the patient's eyebrows are clearly visible
- + The face support must not encroach into the periorbital area above or below the eye
- + Recheck the head support after turning the patient and at least every 30 minutes throughout the procedure
- + It may be necessary to lift and replace the patient's face during long procedures or if there has been movement

See the instructional video at [www.care-surgical.com](http://www.care-surgical.com)

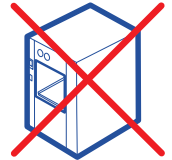
## Cleaning Instructions

Clean using standard hospital wipes

Do not immerse in water

Do not autoclave

UK: Care Surgical Ltd  
Unit 6 Ringtail Road,  
Burscough,  
L40 8JY  
Tel: 01704 336671

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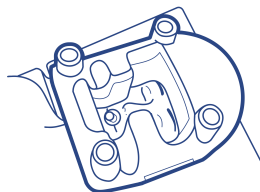
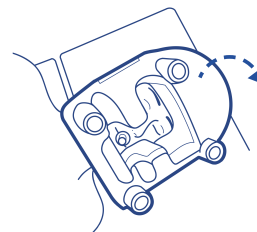
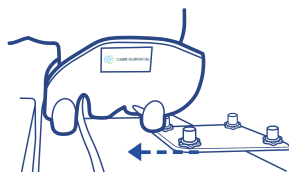
UK : Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel. : 01704 336671

USA : Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel. : 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,  
Lucan Co. Dublin, K78 X5P8 Ireland

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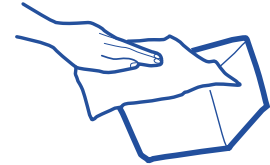
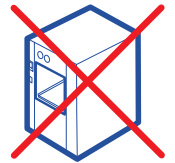
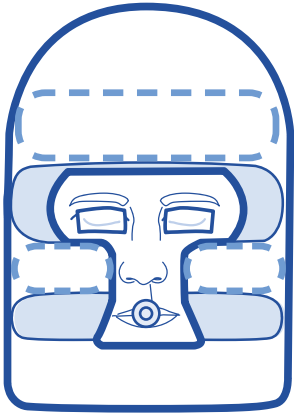
## CS Prone Head Support System (Helmet & Mirror)

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