





⚠ Only to be used by trained clinicians. Ensure product is secure before positioning patient.

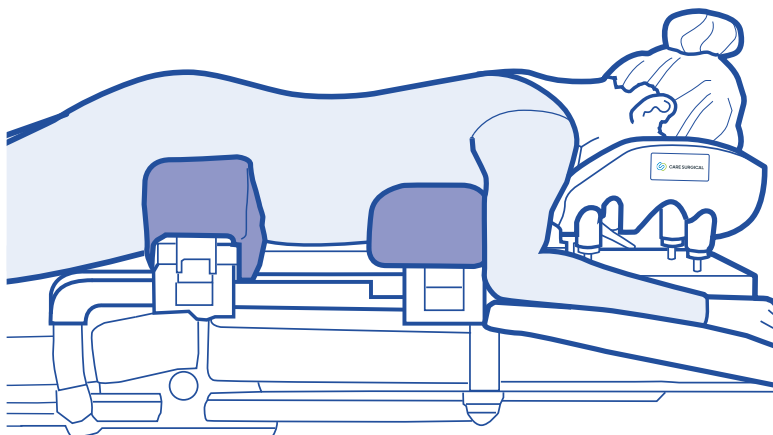

Components


- 1x Spine Frame™ (CSM2712)
- * 2x CS Spine Frame Long Strap (Silicone) (CSM2712-10)
- 4x CS Spine Frame Short Strap (Silicone) (CSM1393)
- 1x Set Disposable Comfort Covers (CSM2606-1)
- 1x Chest Support (CSM2709)
- 1x Hip Supports (CSM2717)

*Please follow the assembly instructions for correct installation of the desired fixing strap variant before initial use.

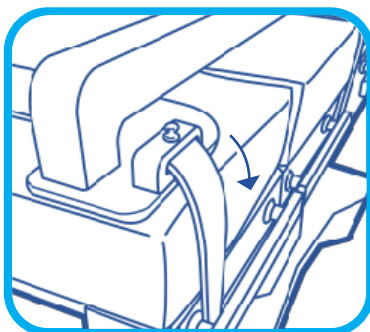


- 1** CSM2712-10: Pass the strap between the table and the side rail once under the tabletop to the other side.

CSM1393: pass the strap between the table and the side rail and loop it around.

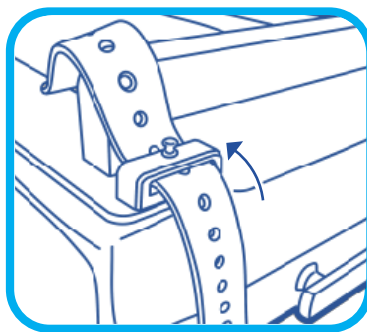


- 2** CSM2712-10: Guide the strap upwards between the table and the side rail, thread it through the buckle and pull it tight.


CSM1393: Thread it through the buckle and pull it tight.

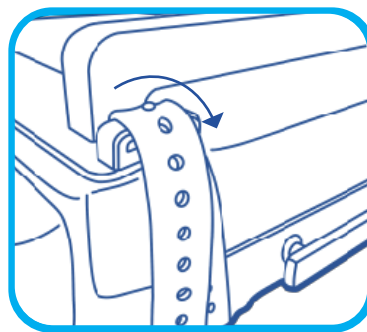


- 3** CSM2712-10 or CSM1393: Secure the strap by pressing the corresponding hole in the strap over the pin, to prevent movement during the procedure.

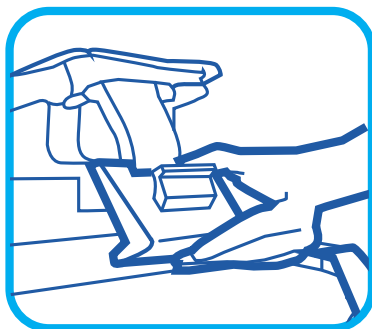

- 4** Place the chest and hip supports on the frame as shown so that the brackets are over the frame.

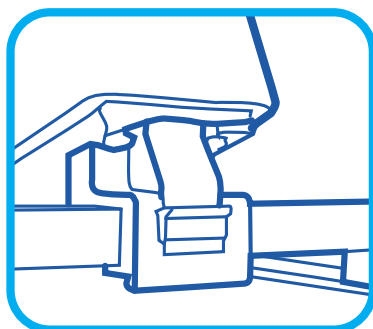

- 5** Push the clip down to lock the support securely in position.

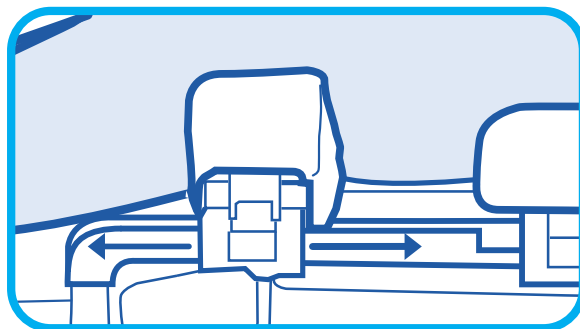

- 6** To adjust the position of the supports, release the clip(s), raise the patients chest or hips, slide the supports to the correct position, re-fasten the clip(s).



UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

EU REP

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland



SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.



DO NOT USE products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.






CARE SURGICAL

Product Code: CSM2715


CS Spine Frame™ Set

Patient positioning

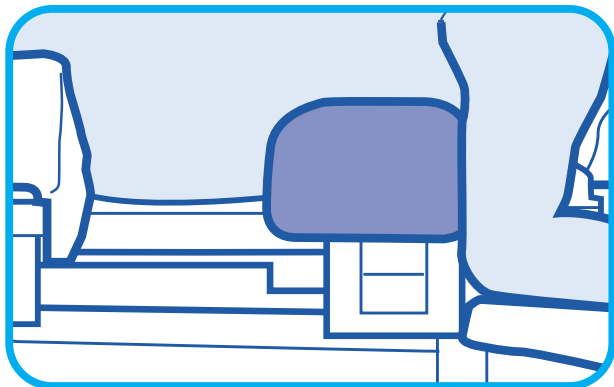


- 1** For male patients, the chest support should be positioned directly under the nipples.

For female patients, move the chest support towards the head to allow easier positioning of the breasts.

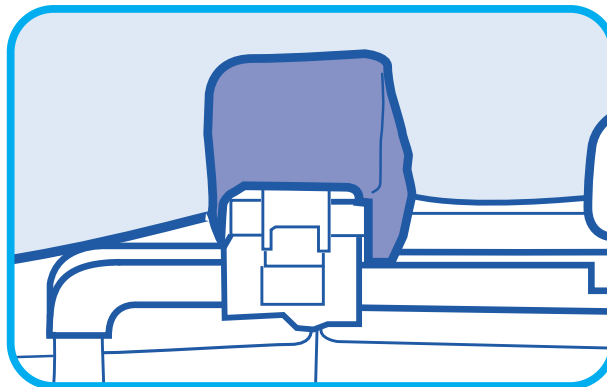

- 2** Ensure the iliac crest is in the centre of the hip pads.


Cleaning Instructions

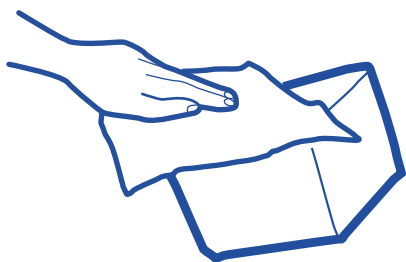

Clean using standard hospital wipes



Do not immerse in water

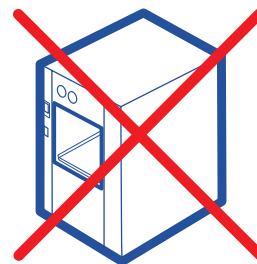

Do not autoclave



UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

EU REP

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland



SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.



DO NOT USE products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.

MD

2

LATEX

NON STERILE

UK CA

CE

250kg
Maximum Patient Weight



Ref: CSM2715 - IFU (Rev.13) 07/2026