




CARE SURGICAL


Product Code: CSM2717

CS Hip Supports (pair)


Patient supports for use on the CS Spine Frame or spinal operating tables

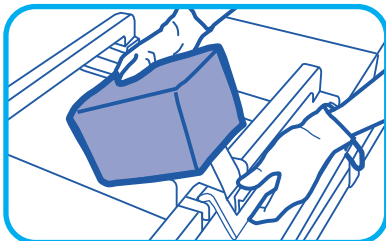

- 1** Place the support on to the frame so that the brackets are over the bars.

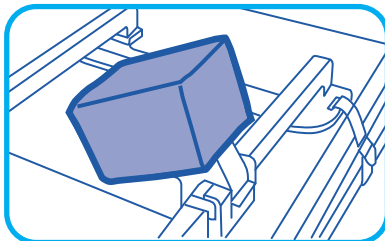

- 2** Snap the locking clip shut to secure the support into position.

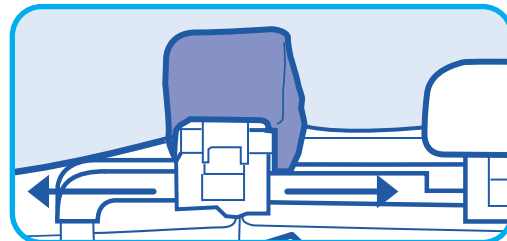

- 3** To adjust the position of the supports, release the clip(s), and slide the supports to the correct position, then snap lock the clip(s)

Ensure the iliac crest is in the centre of the hip pads. Do not exceed the weight capacity of the operating table. Compatible with rail sizes that measure 1.25" (3.2cm) wide and 1.5" (3.8 cm) tall Refer to the label on the support to ensure it is placed in the correct position on the table.

Cleaning instructions:



Clean with standard hospital wipes



Do not immerse in water



Do not autoclave

UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107



Maximum Patient Weight

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland



SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.
DO NOT USE products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.



5 060624 120657

Ref: CSM2717 - IFU (Rev.09) 03/2025

EU REP