



Technical Data Sheet

CSM-2718 CS Thigh Supports (Pair)



Description

- Supports the patient's thighs in the prone position
- Attach to the CS Spine Frame or other suitable modular operating frame or table to support the patient's thighs whilst in the prone position.
- Use the incorporated clips to attach the support to the table
- Open the clip to slide the support laterally into the required position
- Close the clip securely to ensure the support does not move during the procedure
- Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)

Application Area

- OR
- Anaesthesia

Measurements

- Length 232mm/9.13"
- Width 158mm/6.22"
- Height 240mm/9.45"

Material Composition

- Polyurethane foam
- Wipeable fabric covering
- Transfer coated PU with anti-bacterial treatment on knitted polyester

Cleaning

- Clean with standard hospital wipes

Order Information

| Product Code | Product Name | Qty |
|--------------|--------------------------|--------|
| CSM-2718 | CS Thigh Supports (Pair) | 1 Pair |

Packaging Information

- Each pair is packaged in a recyclable polyethylene bag
- 1 pair is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top
- Avoid storing where sharp objects could pierce the skin of the product

Contraindications

Incorrect positioning of the patient on this product can cause:

- Pressure marking or ulcers
- Neurological damage, which can cause paraplegia
- Venous pressure, which can lead to ischaemia

Examine the pad for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced

- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel
- If the clips are not fully closed, the support could move during the procedure which could cause injury to the patient or clinician

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA: Y
- MD Class: 1
- Basic UDI/GTIN No: 5060624120633 (Left), 5060624121081 (Right)

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
Marlborough
MA 01752
United States
Tel: +1-866-243-4107

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Rev.02 04/03/2024