



Technical Data Sheet

CSM2565-1 CS Prone Plus Helmet



Description

- Helmet for use in the prone position
- The helmet holds the CS Prone Plus Face Cushion, CS Prone Plus Cushion or CS Small Face Cushion to maintain its integrity throughout prone procedures
- Space for safe positioning of ET tubes
- For use with the CS Adjustable Mirror
- The patient can be turned from the supine to the prone position without the ET tubes being disconnected when the system is used with the CS Prone Face Cushion

Application Area

- OR
- Anaesthesia

Measurements

- Length: 317mm/12.48"
- Width: 233mm/9.17"
- Height: 155mm/6.10"

Material Composition

- Made with ABS Plastic
- Latex & phthalate free

Cleaning

- Clean the helmet with non-alcohol wipes or warm soapy water

Order Information

Product Code	Product Name	Qty
CSM2565-1	CS Prone Plus Helmet	1 Unit

Packaging Information

- Each unit is packaged in a recyclable polyethene bag
- 1 unit is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top

Contraindications

Incorrect positioning of the patient on this product can cause:

- Pressure marking or ulcers
- POVL
- Neurological damage, which can cause paraplegia
- Venous pressure, which can lead to ischaemia

This product should not be used if:

- There is a chip anywhere on the helmet with a sharp edge
- There is a crack in the helmet
- The shape of the helmet has become compromised/warped. This could cause lack of stability when placed on the mirror or table, or the Face Cushion not to sit squarely within the helmet

- Do not autoclave
- Only to be used by trained medical personnel

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA: Y
- MD Class: 1
- UDI-DI/GTIN No: 5060624120169

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson
MA 01749, USA
Tel: +1-866-243-4107

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Australian Sponsor:
MDSS AU Pty Ltd, Ground Floor, Suite 55,
97-99 Bathurst Street, Sydney, NSW 2000, Australia



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity
Assessed



CE Mark