



Technical Data Sheet

CSM2715 CS Spine Frame™ Set



Order Information

Product Code	Product Name	Qty
CSM2715	CS Spine Frame™ Set	1 Set

Description

- CSM2715 includes:
- CSM2709 CS Chest Support
- CSM2717 CS Hip Supports (Pair)
- CSM2712 CS Spine Frame™
- CSM2712-4 CS Spine Frame™ Straps (set of 4)
- CSM2606-1: CS Comfort Covers - 3 Piece (3 Sets Included)
- Supports patients in the prone position
- Reusable: Clean using non-alcohol hospital wipes
- Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)

Application Area

- OR
- Anaesthesia

Measurements

CSM2709

- Length: 208mm/8.19"
- Width: 480mm/18.9"
- Height: 175mm/6.89"

CSM2717

- Length: 210mm/8.27"
- Width: 136mm/5.35"
- Height: 230mm/9.06"

CSM2712

- Length: 650mm/26"
- Width: 510mm/5.35"
- Height: 67/9.53"

Material Composition

- Wipeable skin covering
- Polyurethane foam
- Transfer coated polyurethane knitted polyester
- Nylon
- Stainless Steel
- Carbon Fibre
- Aluminium
- Woven Polyester

Packaging Information

- Each set is packaged in a recyclable polyethylene bag
- 1 set is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top
- Avoid storing where sharp objects could pierce the skin of the product

Contraindications

Incorrect positioning of the patient on this product can cause:

- Pressure marking or ulcers
- Neurological damage, which can cause paraplegia
- Venous pressure, which can lead to ischaemia

Examine the pad for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced.

- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel
- If the clips are not fully closed, the support could move during the procedure which could cause injury to the patient or clinician

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/UKCA: Y
- MD Class: 1
- UDI-DI/GTIN No:
 - CSM2709: 5060624120046
 - CSM2717: 5060624120756
 - CSM2712: 5060624120763
 - CSM2712-4: 5060624120770
 - CSM2606-1: 5060624121166

Cleaning

- Clean with standard hospital wipes

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson
MA 01749, USA
Tel: +1-866-243-4107

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark



Technical Data Sheet

CSM2709 CS Chest Support



Description

- Supports the patient's chest in the prone position
- Attach to the CS Spine Frame or other suitable modular operating frame or table to support the patient's chest whilst in the prone position
- Use the incorporated clips to attach the support to the table
- Open the clip to slide the chest support laterally into the required position
- Close the clip securely to ensure the support does not move during the procedure
- Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)

Application Area

- OR
- Anaesthesia

Measurements

- Length: 208mm/8.19"
- Width: 480mm/18.90"
- Height: 175mm/6.89"

Material Composition

- Wipeable skin covering
- Polyurethane foam
- Transfer coated PU with anti-bacterial treatment on knitted polyester

Cleaning

- Clean with standard hospital wipes

Order Information

Product Code	Product Name	Qty
CSM2709	CS Chest Support	1 Unit

Packaging Information

- Each unit is packaged in a recyclable polyethylene bag
- 1 unit is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top
- Avoid storing where sharp objects could pierce the skin of the product

Contraindications

Incorrect positioning of the patient on this product can cause:

- Pressure marking or ulcers
- Neurological damage, which can cause paraplegia
- Venous pressure, which can lead to ischaemia

Examine the pad for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced.

- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel
- If the clips are not fully closed, the support could move during the procedure which could cause injury to the patient or clinician

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA: Y
- MD Class: 1
- UDI-DI/GTIN No: 5060624120046

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson
MA 01749, USA
Tel: +1-866-243-4107



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Australian Sponsor:

MDSS AU Pty Ltd, Ground Floor, Suite 55,
97-99 Bathurst Street, Sydney, NSW 2000, Australia

Ref: CSM2709 - TDS (Rev.07) 03/2026



Technical Data Sheet

CSM2717 CS Hip Support (Pair)



Description

- Supports the patient's hip in the prone position
- Attach to the CS Spine Frame or other suitable modular operating frame or table to support the patient's hip(s) whilst in the prone position
- Use the incorporated clip to attach the support to the table
- Open the clip to slide the chest support laterally into the required position
- Close the clip securely to ensure the support does not move during the procedure
- Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)

Application Area

- OR
- Anaesthesia

Measurements

- Length: 210mm/8.26"
- Width: 136mm/5.35"
- Height: 230mm/9.05"

Material Composition

- Wipeable skin covering
- Polyurethane foam
- Transfer coated PU with anti-bacterial treatment on knitted polyester

Cleaning

- Clean with standard hospital wipes

Order Information

Product Code	Product Name	Qty
CSM2717	CS Hip Support (Pair)	1 Unit

Packaging Information

- Each unit is packaged in a recyclable polyethylene bag
- 1 unit is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top
- Avoid storing where sharp objects could pierce the skin of the product

Contraindications

Incorrect positioning of the patient on this product can cause:

- Pressure marking or ulcers
- Neurological damage, which can cause paraplegia
- Venous pressure, which can lead to ischaemia

Examine the pad for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced.

- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel
- If the clips are not fully closed, the support could move during the procedure which could cause injury to the patient or clinician

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA: Y
- MD Class: 1
- UDI-DI/GTIN No: 5060624120756

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson
MA 01749, USA
Tel: +1-866-243-4107



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Australian Sponsor:

MDSS AU Pty Ltd, Ground Floor, Suite 55,
97-99 Bathurst Street, Sydney, NSW 2000, Australia

Ref: CSM2717 - TDS (Rev.07) 03/2026



Technical Data Sheet

CSM2712 CS Spine Frame™



Description

- Use with related components to support patients in the prone position
- Part of a set that converts a standard operation table into a modular spinal table
- Reusable: Clean using non-alcohol hospital wipes
- Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)

Application Area

- OR
- Anaesthesia

Measurements

- Length: 650mm/26"
- Width: 510mm/5.35"
- Height: 67/9.53"

Material Composition

- Nylon
- Stainless Steel
- Carbon Fibre

Cleaning

- Clean with standard hospital wipes

Order Information

Product Code	Product Name	Qty
CSM2712	CS Spine Frame™	1 Unit

Packaging Information

- Each frame is packaged in a recyclable polyethene bag
- 1 frame is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight

Contraindications

- Only use with recommended Care Surgical components and accessories
- Ensure the frame is securely attached to the table by 4 CS Spine Frame™ straps before use
- Check the straps and buckles for damage before and after each use. Do not use if damaged. Contact supplier for advice.
- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/UKCA: Y
- MD Class: 1
- UDI-DI/GTIN No: 5060624120763

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson
MA 01749, USA
Tel: +1-866-243-4107

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Ref: CSM2712 - TDS (Rev.07) 03/2026



Technical Data Sheet

CSM2712-4 CS Spine Frame™ Straps (set of 4)



Description

- Set of 4 webbing (polyester) straps 500x40mm with plastic (Polyoxymethylene) buckles
- For use with CSM2712 CS Spine Frame
- Secure the CS Spine Frame to the side bars of the operating table using the 4 straps provided
- Instructions for use are packaged with the product

Application Area

- OR
- Anaesthesia

Measurements

- Length: 540mm/21.2"
- Width: 50mm/1.9"
- Height: 10mm/0.39"

Material Composition

- 4 webbing (polyester) straps
- Plastic (Polyoxymethylene) buckles

Cleaning

- Clean with standard hospital wipes

Order Information

Product Code	Product Name	Qty
CSM2712-4	CS Spine Frame™ Straps (set of 4)	1 Set

Packaging Information

- Each set is packaged in a recyclable polyethene bag
- 1 set is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top
- Avoid storing where sharp objects could pierce the skin of the product

Contraindications

- Ensure the 4 straps are tightened evenly
- If the straps are too loose, it could cause the frame to move during use
- If the straps are too tight, it could cause damage to the straps, the frame or the operating table
- Check the straps and buckles for damage before and after each use. Do not use if damaged
- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/UKCA:Y
- MD Class: 1
- UDI-DI/GTIN No: 5060624120770

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson
MA 01749, USA
Tel: +1-866-243-4107

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark