




Technical Data Sheet

CSM2717 CS Hip Support (Pair)

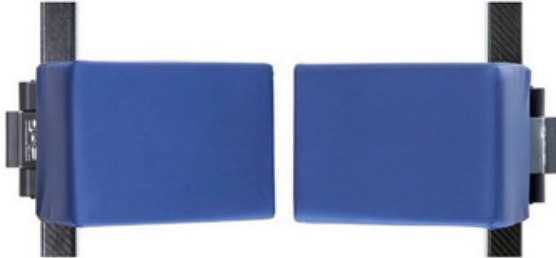

Description

- Supports the patient's hip in the prone position
- Attach to the CS Spine Frame or other suitable modular operating frame or table to support the patient's hip(s) whilst in the prone position
- Use the incorporated clip to attach the support to the table
- Open the clip to slide the chest support laterally into the required position
- Close the clip securely to ensure the support does not move during the procedure
- Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)

Application Area

- OR
- Anaesthesia

Measurements

- Length: 210mm/8.26"
- Width: 136mm/5.35"
- Height: 230mm/9.05"

Material Composition

- Wipeable skin covering
- Polyurethane foam
- Transfer coated PU with anti-bacterial treatment on knitted polyester

Cleaning

- Clean with standard hospital wipes

Order Information

Product Code	Product Name	Qty
CSM2717	CS Hip Support (Pair)	1 Unit

Packaging Information

- Each unit is packaged in a recyclable polyethylene bag
- 1 unit is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top
- Avoid storing where sharp objects could pierce the skin of the product

Contraindications

Incorrect positioning of the patient on this product can cause:

- Pressure marking or ulcers
- Neurological damage, which can cause paraplegia
- Venous pressure, which can lead to ischaemia

Examine the pad for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced.

- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel
- If the clips are not fully closed, the support could move during the procedure which could cause injury to the patient or clinician

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA: Y
- MD Class: 1
- UDI-DI/GTIN No: 5060624120756

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson
MA 01749, USA
Tel: +1-866-243-4107



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Australian Sponsor:

MDSS AU Pty Ltd, Ground Floor, Suite 55,
97-99 Bathurst Street, Sydney, NSW 2000, Australia

Ref: CSM2717 - TDS (Rev.07) 03/2026