



Technical Data Sheet

CSM2821 CS Knee Rings (pair)



Description

- Supports the patients' knees in the prone position
- Reduces the risk of pressure ulcers

Application Area

- OR
- Anaesthesia

Measurements

- Length: 230mm/ 9.05"
- Width: 230mm/ 9.05"
- Height 60 mm/ 2.36 "

Material Composition

- Polyurethane foam core
- Latex & phthalate free
- Transfer coated PU with anti-bacterial treatment on knitted polyester

Cleaning

- Clean with standard hospital wipes

Order Information

Product Code	Product Name	Qty
CSM2821	CS Knee Rings	1 Pair

Packaging Information

- Each pair is packaged in a recyclable polyethylene bag
- 1 pair is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top
- Avoid storing where sharp objects could pierce the skin of the product

Contraindications

Incorrect positioning of the patient on this product can cause:

- Pressure marking or ulcers
- Neurological damage, which can cause paraplegia;
- Venous pressure, which can lead to ischaemia.

Examine the pad for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced.

- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/UKCA:Y
- MD Class: 1
- UDI-DI/GTIN No: 5060624121128

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson
MA 01749, USA
Tel: +1-866-243-4107

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Australian Sponsor:

MDSS AU Pty Ltd, Ground Floor, Suite 55,
97-99 Bathurst Street, Sydney, NSW 2000, Australia



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Ref: CSM2821 - TDS (Rev.03) 03/2026