




CARE SURGICAL

# CS Comfort Covers Large - 5 Piece

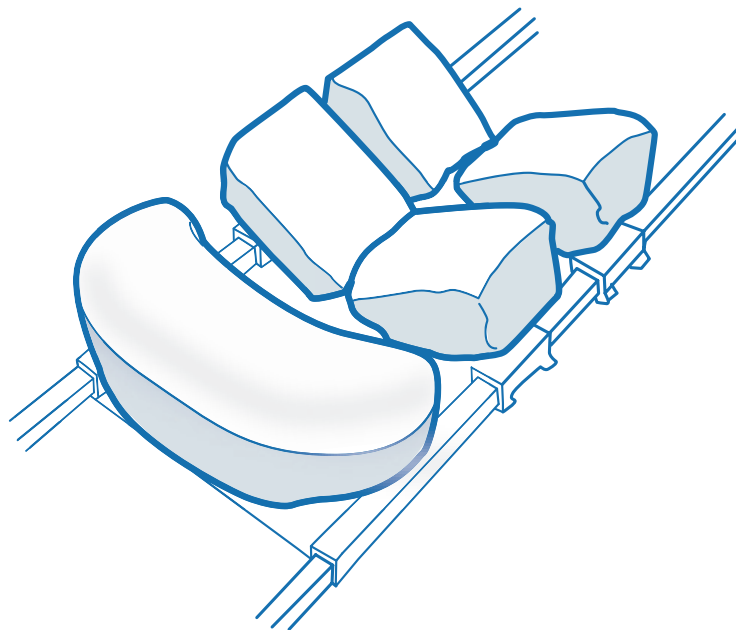

**CS Comfort Covers Large**  
**2 Large Hip Covers**  
**2 Thigh Covers**  
**1 Chest Cover**

Enhances infection control and protects the pads from damage. Reduces the chance of pressure related injury.



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

**www.care-surgical.com**

Care Surgical Supplies Limited, Office 2, 12a Lower Main  
Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity  
Assessed



Do Not Reuse



CE Mark



Latex Free



Non Sterile



5 060624 121203



LOT:

EC REP

Product Code: CSM-2603 Rev.3 02/24




## Head-to-Toe Check list

This check should be completed once the patient is in the prone position and before surgical drapes are applied. At all stages, check that no tubing is trapped between the patient and the supports, and that all bony protrusions are padded.

### The head

- + Eyes lightly taped, no padding or goggles
- + Face supported on the cheeks and forehead
- + No pressure on the nerves around the eyes
- + No pressure or minimal pressure on the chin

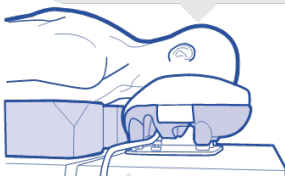


### Neck

- + Chin and throat not touching the chest support
- + Natural, neutral neckline

### Chest, abdomen and pelvis

- + ECG electrodes positioned on the patient's back
- + Breast positioned medically with minimum pressure
- + Adequate space to avoid compression of the abdomen
- + Male genitalia free of the pelvic support

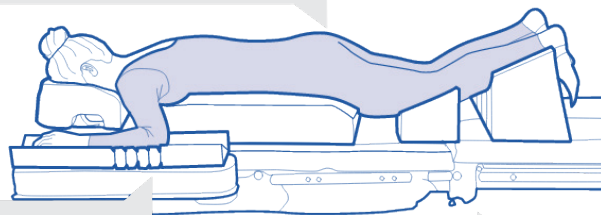

### The arms

- + Arms forward and not abducted beyond 90° or arms secure by the patient's side
- + No pressure on the axillary nerve
- + No body weight supported on the arms

### Legs and feet

- + No pressure on the thighs
- + Ankles raised

To see our full range of Care Surgical Medical Products, visit [www.care-surgical.com](http://www.care-surgical.com)



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107



Medical Device



UK Conformity  
Assessed



Do not reuse



CE Mark



Latex Free



Non Sterile



[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,  
Lucan Co. Dublin, K78 X5P8 Ireland