





CARE SURGICAL

CS Bow & Wilson Comfort Covers


Covers to protect the Wilson or Allen Bow® Frame from damage and to aid infection control

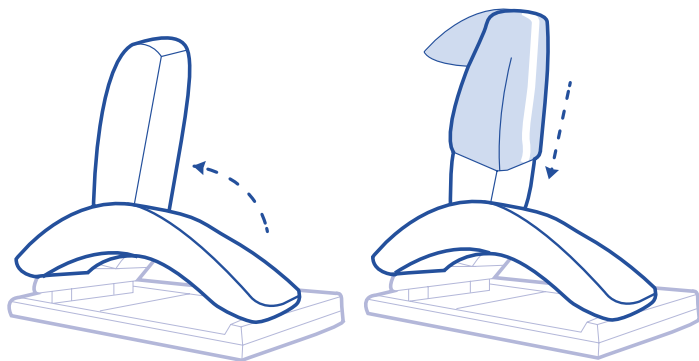

- 1** Raise the arch of the Frame and slide the cover onto it. Ensure the white band is on the underside of the arch so the patient does not come in contact with the seam.

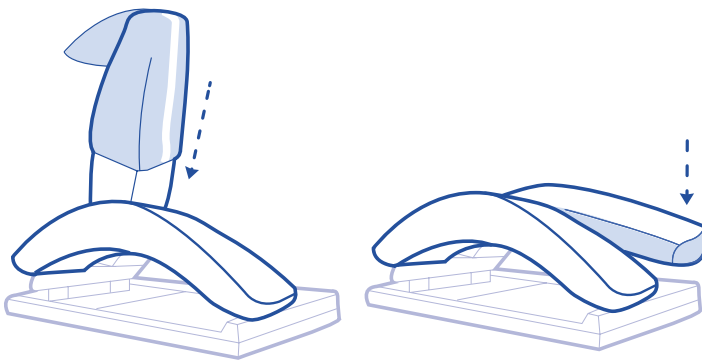

- 2** Return the arch to its position and repeat with the other arch.

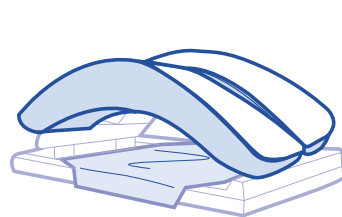

- 3** Use the square drape to protect the base of the frame. Slide the bar roll onto the centre bar of the frame if applicable.

Consult the instructions for the Wilson Frame for guidance on correct positioning of the patient.



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main
Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity
Assessed



Do not reuse



CE Mark



Latex Free



Non Sterile




LOT:

EC REP

Product Code: CSM-2608 Rev.9 02/24

Compatible Products

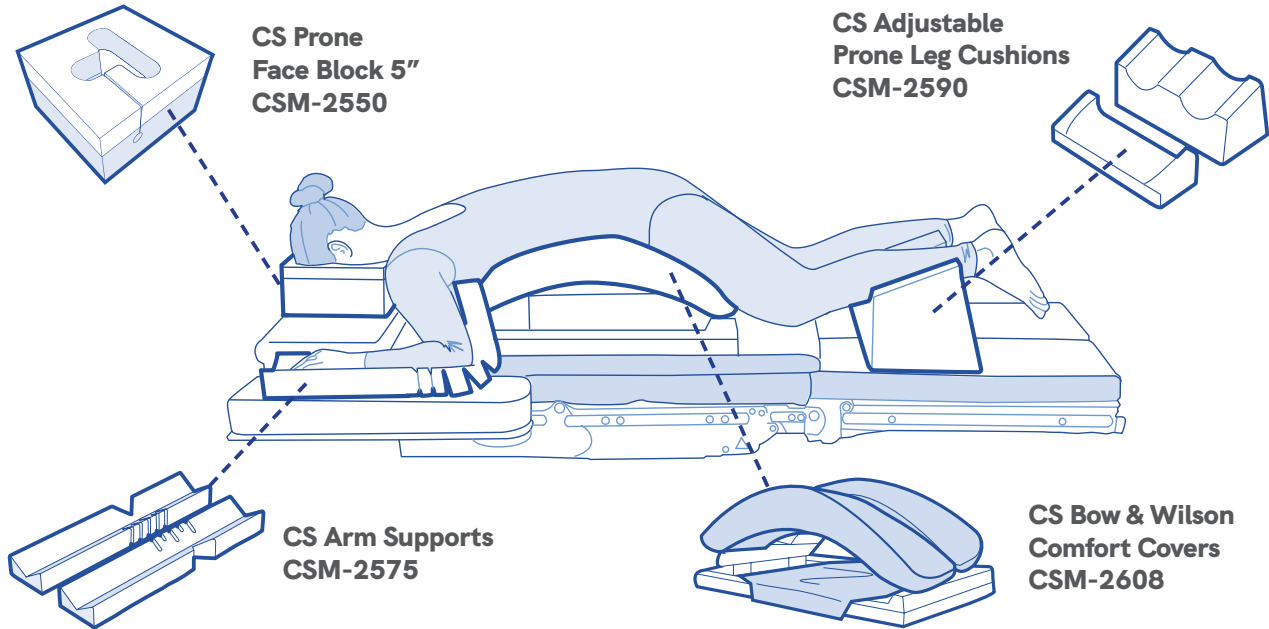

To see the full range of Care Surgical Products, visit www.care-surgical.com



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main
Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity
Assessed



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120244

EC REP

ProductCode: CSM-2608Rev.9 02/24