



CARE SURGICAL


Product Code: CSM2709

CS Chest Support

(CSM2709-09 CS Chest Pad & CSM2709-10 CS Chest Bracket)

 Only to be used by trained clinicians. Ensure product is secure before positioning patient.

Patient Supports for use on the CS Spine Frame or spinal operating tables

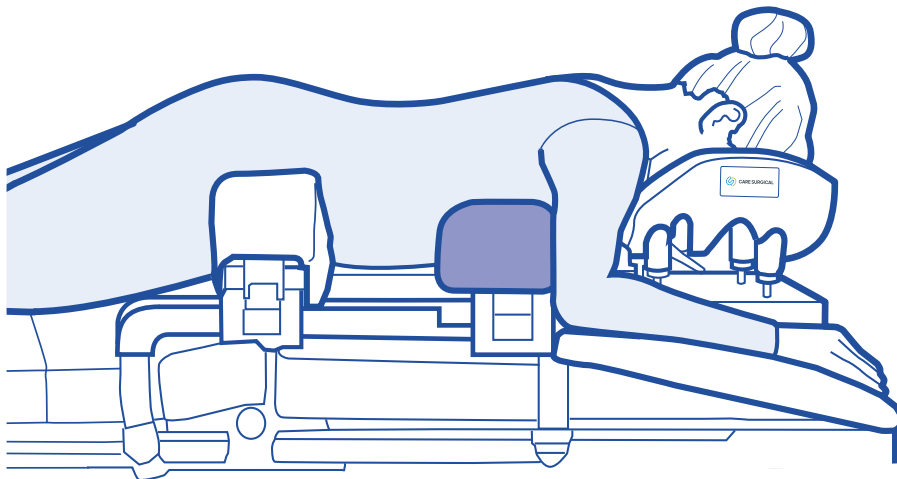

Refer to the label on the support to ensure it is placed in the correct position on the table.



UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107



www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland



SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.


DO NOT USE products that appear to be damaged or worn.

DO NOT USE products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.




Ref: CSM2709 - IFU (Rev.11) 06/2025


EU REP

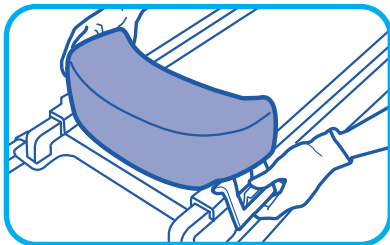

- 1** Place the support on to the frame so that the brackets are over the bars.

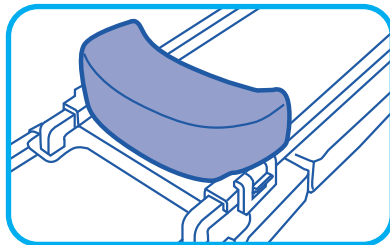

- 2** Snap the locking clip shut to secure the support into position.

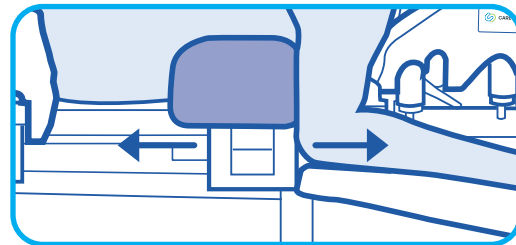

- 3** To adjust the position of the supports, release the clip(s), and slide the supports to the correct position, then snap lock the clip(s)

For male patients, the chest support should be positioned directly under the nipples.
For female patients, move the chest support towards the head to allow easier positioning of the breast.
Do not exceed the weight capacity of the operating table.

Cleaning instructions:



Clean using standard hospital wipes



Do not immerse in water



Do not autoclave



UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107



www.care-surgical.com



Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

Ref: CSM2709 - IFU (Rev.11) 06/2025