




# CARE SURGICAL

## CS Head Tray Attachment

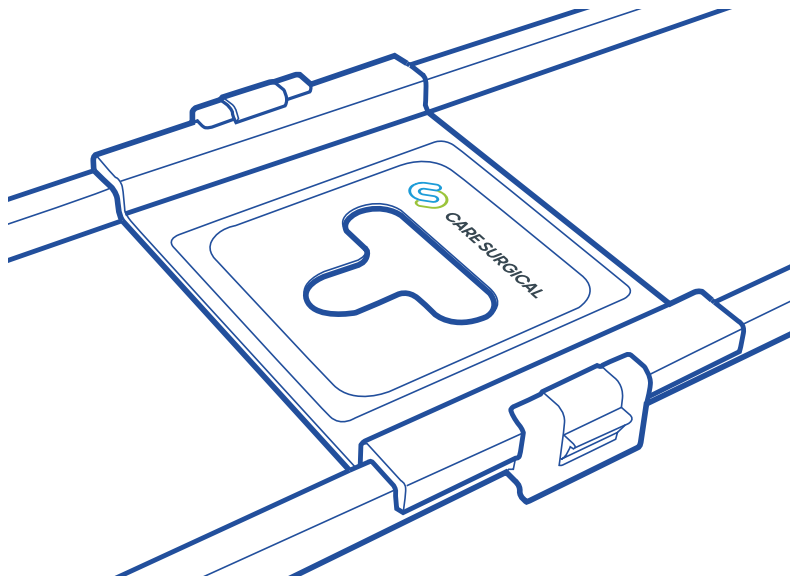

**CS Head Tray Attachment**  
Compatible with all Allen and Jackson spinal tables.

Head tray attachment for use with the  
CS Prone Face Block, CS Prone Head  
Support System or CS Prone Plus  
Head Support System



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

**www.care-surgical.com**

EC REP

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,  
Lucan Co. Dublin, K78 X5P8 Ireland



5 060624 121098



LOT:



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity  
Assessed



CE Mark




Maximum  
Weight

Product Code: CSM-2714 Rev.5 02/24




# CARE SURGICAL


## CS Head Tray Attachment

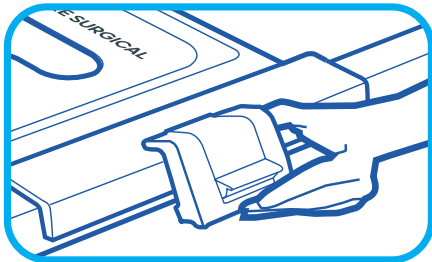

- 1** Place the attachment on the frame so that the brackets are over the bars

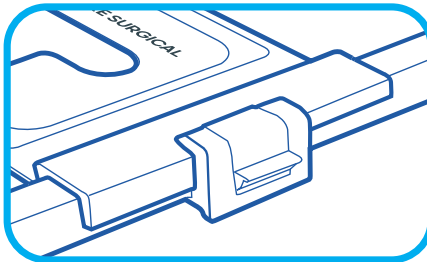

- 2** Snap the locking clip shut to secure the attachment in position

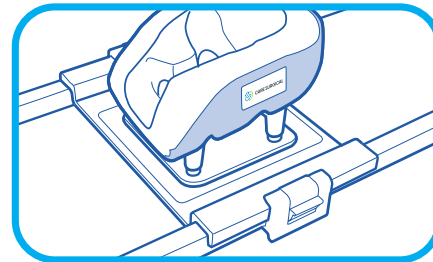

- 3** Place prone head support in the centre of the attachment

### Cleaning instructions:



Clean using standard hospital wipes



Do not immerse in water



Do not autoclave



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)



Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,  
Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity  
Assessed



CE Mark



Maximum  
Weight



5 060624 121098

Product Code: CSM-2714 Rev.5 02/24