



## Components


- 1 Spine Frame, 4 x Straps
- 1 x Set Disposable Comfort Covers
- 1 x Chest Support, 2 x Hip Supports

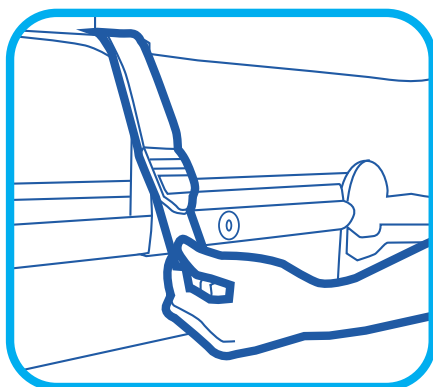


- 1** Loop the straps under the side bars of the table and through the buckles.

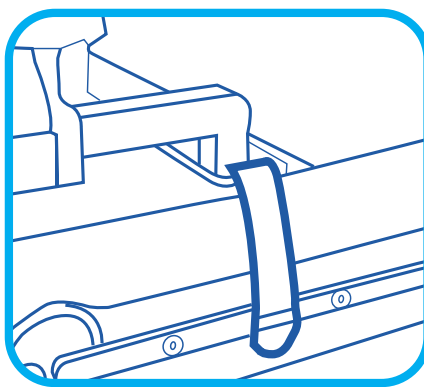


- 2** Tighten the straps until the frame is firmly attached to the table to prevent movement during the procedure.

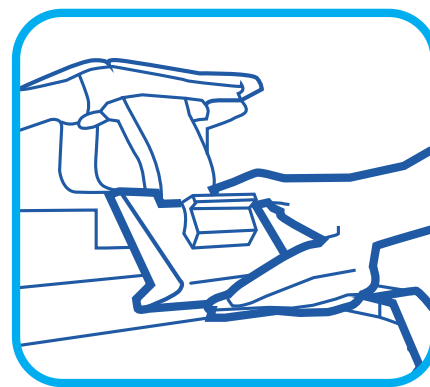

- 3** Place the chest and hip supports on the frame as shown so that the brackets are over the frame.

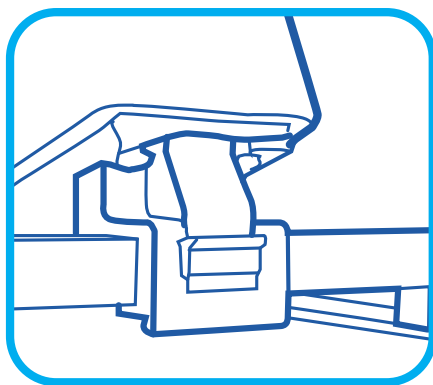

- 4** Push the clip down to lock the support securely in position.

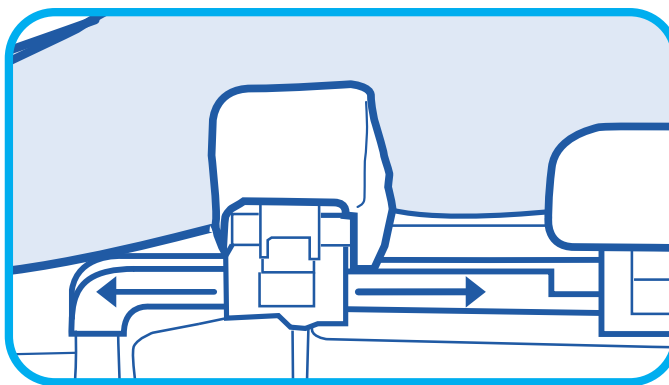

- 5** To adjust the position of the supports, release the clip(s), raise the patient's chest or hips, slide the supports to the correct position, re-fasten the clip(s).



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)



Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,  
Lucan Co. Dublin, K78 X5P8 Ireland



5 060624 120596



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity  
Assessed



CE Mark




Maximum  
Weight



LOT:




## Patient positioning

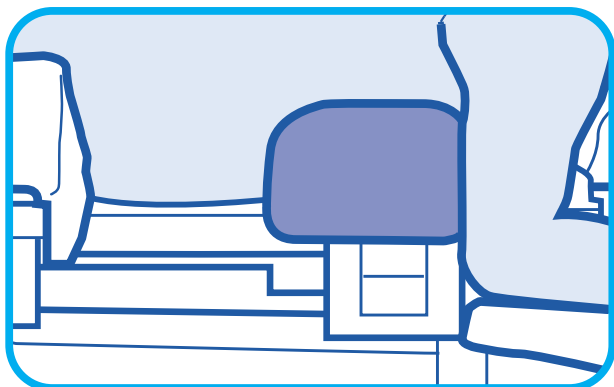


- 1** For male patients, the chest support should be positioned directly under the nipples.

For female patients, move the chest support towards the head to allow easier positioning of the breasts.

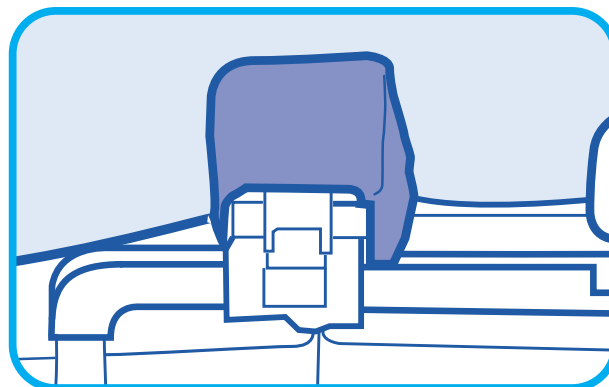

- 2** Ensure the iliac crest is in the centre of the hip pads.


## Cleaning Instructions

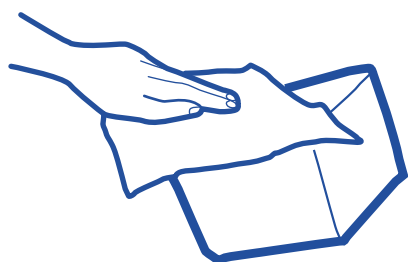

Clean using standard hospital wipes



Do not immerse in water

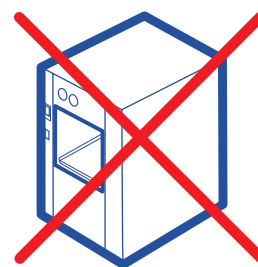


Do not autoclave



UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

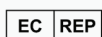

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,  
Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity  
Assessed



CE Mark



Maximum  
Weight



5 060624 120596