




CARE SURGICAL

# CS Comfort Shin Supports (Pair)

## Instructions for use

- + Ensure feet do not touch operating table
- + Ankles should extend beyond Shin support

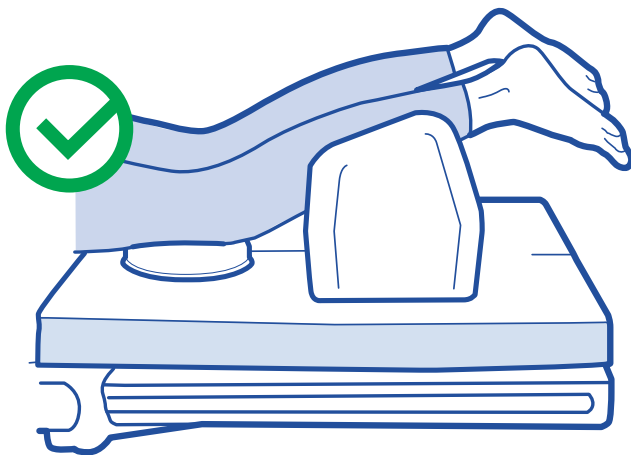

Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,  
Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity  
Assessed



Reuse



CE Mark



Latex Free



Non Sterile



5 060624 120725

LOT Number:

EC REP


Product Code: CSM-2820 Rev.6 09/23

# Head-to-Toe Check list

This check should be completed once the patient is in the prone position and before the surgical drapes are applied. At all stages, check that no tubing is trapped between the patient and the supports, and that all bony protrusions are padded.

## The head

- + Eyes lightly taped, no padding or goggles
- + Face supported on the cheeks and forehead
- + No pressure on the nerves around the eyes
- + No pressure or minimal pressure on the chin

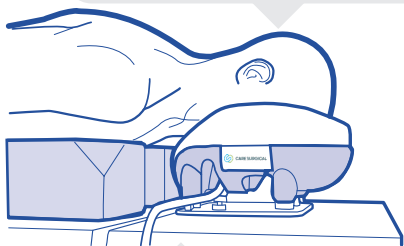


## Neck

- + Chin and throat not touching the chest support
- + Natural neutral neckline

## Chest, abdomen and pelvis

- + ECG electrodes positioned on the patient's back
- + Breasts positioned medially with minimum pressure
- + Adequate space to avoid compression of the abdomen
- + Male genitalia free of the pelvic support

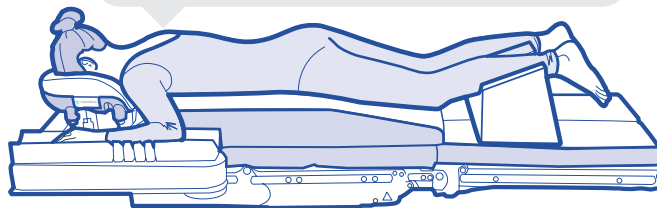

## The arms

- + Arms forward and not abducted beyond 90°, or arms secure by the patient's side
- + No pressure on the axillary nerve
- + No body weight supported on the arms

To see the full range of Care Surgical Medical Products, visit [www.care-surgical.com](http://www.care-surgical.com)



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107



Medical Device UK Conformity Assessed



Reuse



CE Mark



Latex Free



Non Sterile



5 060624 120725

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,  
Lucan Co. Dublin, K78 X5P8 Ireland

EC REP

Product Code: CSM-2820 Rev.6 09/23