




CARE SURGICAL

CS Knee Rings (Pair)

Instructions for use

- + Place the knees in the centre of the rings for minimum pressure

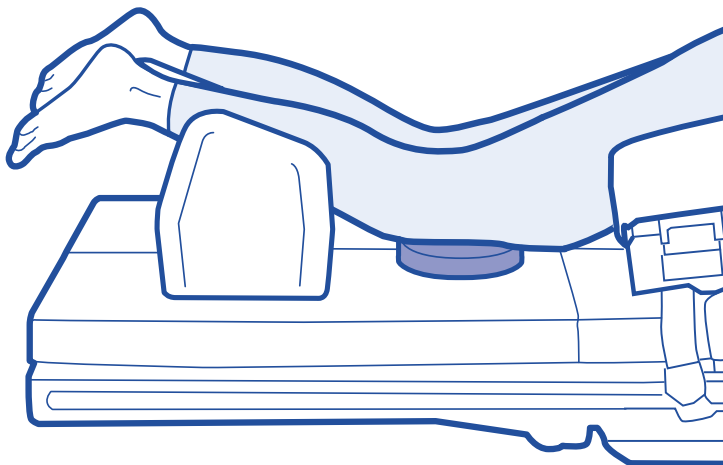


Cleaning instructions:



Clean using standard hospital wipes



Do not immerse in water

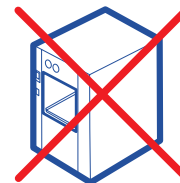

Do not autoclave



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity
Assessed



CE Mark



5 060624 121128



LOT:

EC REP


Product Code: CSM-2821 Rev.6 04/23

Head-to-Toe Check list

This check should be completed once the patient is in the prone position and before the surgical drapes are applied. At all stages, check that no tubing is trapped between the patient and the supports, and that all bony protrusions are padded.

The head

- + Eyes lightly taped, no padding or goggles
- + Face supported on the cheeks and forehead
- + No pressure on the nerves around the eyes
- + No pressure or minimal pressure on the chin

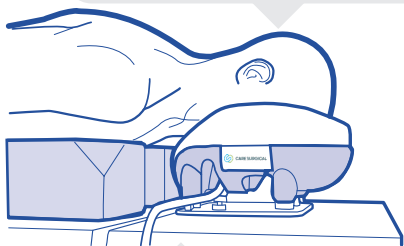


Neck

- + Chin and throat not touching the chest support
- + Natural neutral neckline

Chest, abdomen and pelvis

- + ECG electrodes positioned on the patient's back
- + Breasts positioned medially with minimum pressure
- + Adequate space to avoid compression of the abdomen
- + Male genitalia free of the pelvic support

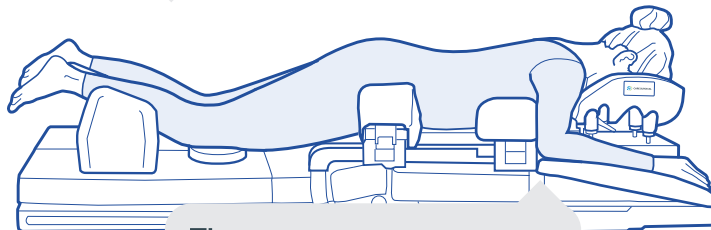

The arms

- + Arms forward and not abducted beyond 90°, or arms secure by the patient's side
- + No pressure on the axillary nerve
- + No body weight supported on the arms

To see the full range of Care Surgical Medical Products, visit www.care-surgical.com



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity
Assessed



CE Mark



5 060624 121128



Product Code: CSM-2821 Rev.6 04/23