




Technical Data Sheet

CSM2550V CS Prone Face Block 5"

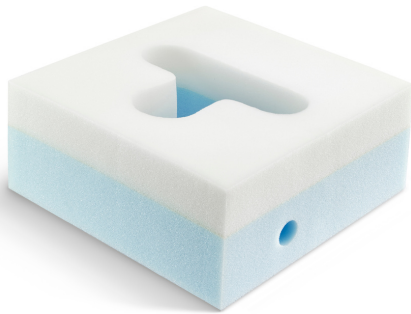

Description

- Soft foam block with slit/hole for ET tube on the right
- Firm base provides stability
- Soft foam top for patient comfort and protection
- Distributes the pressure on the patient's cheeks, forehead and chin while in the prone position
- Raises the patient's head to assist with achieving a natural, neutral neckline
- For patients aged 8 and over, dependent on stature
- For patient's weighing up to 200kg
- Remove from bag 1 (one) hour before use to allow the cushion to regain required shape and size
- Ensure product is fully inflated before use

Application Area

- OR
- Anaesthesia

Measurements

- L240 | W260 | H130 mm
- L9.4 | W10.2 | H5"

Material Composition

- Polyurethane foam
- Latex & phthalate free

Order Information

Product Code	Product Name	Qty
CSM2550V	CS Prone Face Block 5"	14 units

Packaging Information

- Each Unit is packaged in a recyclable polyethene bag (compressed)
- 14 units are packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top

Contraindications

Incorrect positioning of the patient on this product can cause:

- Pressure marking or ulcers
- Neurological damage, which can cause paraplegia
- Venous pressure, which can lead to ischaemia

Further Information

- Disposable/Reusable: Disposable
- Sterility: N
- CE/FDA/TGA/UKCA:Y
- MD Class: 1
- Basic UDI/GTIN No: 5060624120091

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson
MA 01749, USA
Tel: +1-866-243-4107

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia



Medical Device



Do Not Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Rev.02 01/04/2025