



Technical Data Sheet

CSM-2569 CS Contour Support Cradle



Description

- 2-Piece wedge cushion set
- Starburst style adapter to attach the cradle to the Allen C-Flex Head Positioning System
- The cradle holds the CS Prone Plus Face Cushion or CS Prone Face Cushion- Small to maintain their integrity throughout prone procedures.
- Space for safe positioning of ET tubes.
- Clean the helmet with non-alcohol wipes or warm soapy water
- Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)

Application Area

- OR
- Anaesthesia

Measurements

- Length 317mm/12.48"
- Width 233mm/9.17"
- Height 234mm/9.21"

Material Composition

- Made from ABS plastic
- Latex & phthalate free

Cleaning

- Clean the helmet with non-alcohol wipes or warm soapy water

Order Information

Product Code	Product Name	Qty
CSM-2569	CS Contour Support Cradle	1 Unit

Packaging Information

- Each unit is packaged in a recyclable polyethylene bag
- 1 unit is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top

Contraindications

Incorrect positioning of the patient on this product can cause:

- Pressure marking or ulcers
- Neurological damage, which can cause paraplegia
- Venous pressure, which can lead to ischaemia

This product should not be used if:

- There is a chip anywhere on the helmet with a sharp edge
- There is a crack in the helmet
- The shape of the helmet has become compromised/warped. This could cause lack of stability when placed on the mirror or table, or the Face Cushion not to sit squarely within the helmet
- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA: Y
- MD Class: 1
- Basic UDI/GTIN No: 5060624120909

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
Marlborough
MA 01752
United States
Tel: +1-866-243-4107

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Rev.01 11/09/2023