



Technical Data Sheet

CSM-2805 CS Supine Head Support (Adult)



Description

- Reusable
- Supports patient's head in the supine position
- Reduces the incidence of pressure ulcers and marking

Application Area

- OR
- Anaesthesia

Measurements

- Outer Ø200 Inner Ø65 H50 mm
- Outer Ø7.8 Inner Ø2.5 H1.9 "

Material Composition

- Polyurethane foam core
- Latex & phthalate free

Cleaning

- Clean with standard hospital wipes

Order Information

Product Code	Product Name	Qty
CSM-2805	CS Supine Head Support (Adult)	1 Pc

Packaging Information

- Each piece is packaged in a recyclable polyethene bag
- The box quantity depends on order quantity

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top
- Avoid storing where sharp objects could pierce the skin of the product

Contraindications

Incorrect positioning of the patient on this product can cause:

- Pressure marking or ulcers
- Nerve damage

Examine the pad for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced.

- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA:Y
- MD Class: 1
- Basic UDI/GTIN No: 5060624120374

Labelling




Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
Marlborough
MA 01752
United States
Tel: +1-866-243-4107

www.care-surgical.com

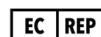

Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

Rev.01 7/23