




# CARE SURGICAL

# CS Tony Prone

## Patient support for use in the safe prone positioning of the conscious or unconscious patient.


Support the unconscious patient in the prone position.

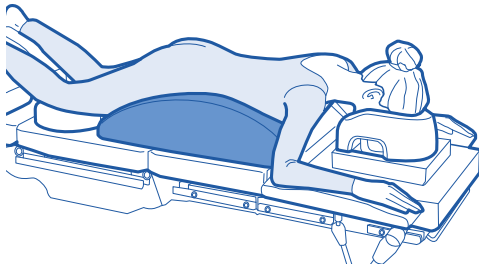

The conscious patient can be supported in the prone position with head placed laterally.



Ensure the CS Tony Prone is positioned with the recess towards the patients hips.

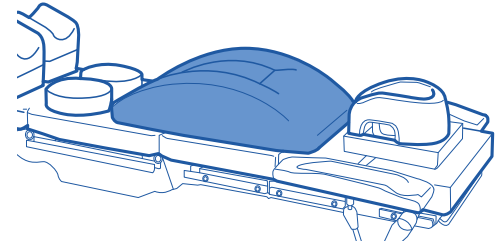

It is recommended that the CS Tony Prone is placed on a firm procedure bed or table. Pillows or supports must be used to support the patient's head and legs.

Only to be used by trained clinicians. Do not exceed weight limit of the supporting table.



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
Marlborough  
MA 01752  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark



Maximum Weight

Australian Sponsor, Summit Surgical  
PO Box 60, Landsborough  
QLD 4550 Australia

LOT Number:


Product Code: CSM-2906 Rev 5 04/22

# Head-to-Toe Check list

This check should be completed once the patient is in the prone position and before the surgical drapes are applied. At all stages, check that no tubing is trapped between the patient and the supports, and that all bony protrusions are padded.


## The head

- + Eyes lightly taped, no padding or goggles
- + Face supported on the cheeks and forehead
- + No pressure on the nerves around the eyes
- + No pressure or minimal pressure on the chin

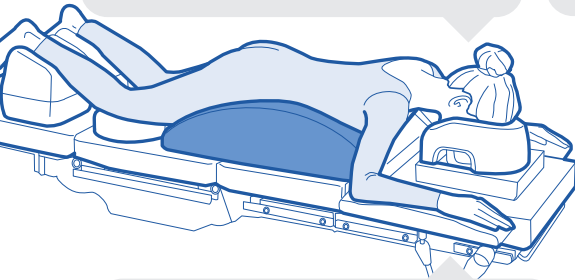

## Chest, abdomen and pelvis

- + ECG electrodes positioned on the patient's back
- + Breasts positioned medially with minimum pressure
- + Adequate space to avoid compression of the abdomen
- + Male genitalia free of the pelvic support

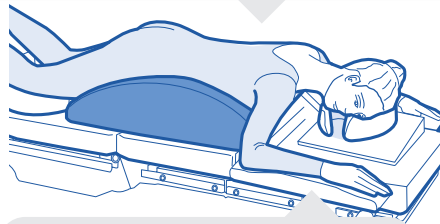


## Neck

- + Chin and throat not touching the chest support
- + Natural neutral neckline

## The arms

- + Arms forward and not abducted beyond 90°, or arms secure by the patient's side
- + No pressure on the axilla nerve
- + No body weight supported on the arms


## Cleaning Instructions

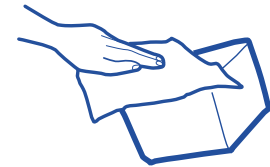

Clean using standard hospital wipes



Do not immerse in water

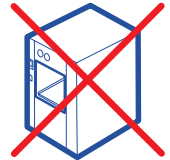

Do not autoclave



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
Marlborough  
MA 01752  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

EC REP



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark



Maximum Weight

Australian Sponsor, Summit Surgical  
PO Box 60, Landsborough  
QLD 4550 Australia

Product Code: CSM-2906 Rev 5 04/22