





# CARE SURGICAL

## CS Armboard Pad- with Strap


### Placing Patient in CS Armboard Pad:

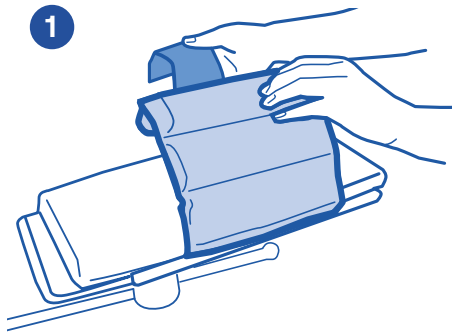

1  
Separate Velcro® strap of Pad from the Velcro® strip on the bottom of the armboard.

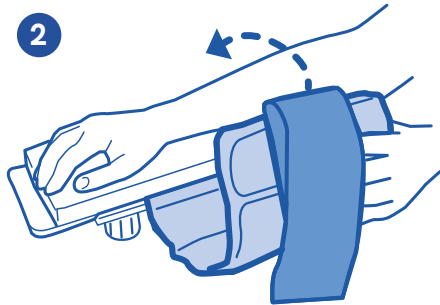

2  
Place patient's arm on top of the pad.

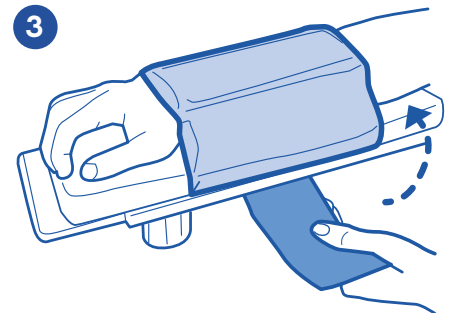

3  
Wrap the shell of the Pad around the patient's arm and secure the Velcro® strap to the bottom of the armboard.



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, Office 2, 12a Lower Main  
Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity  
Assessed



Reuse



CE Mark



Latex Free



Non Sterile



LOT Number:



Product Code: CSM-2576-20 Rev. 6 02/24



# CARE SURGICAL

## Instructions for use

### Pad cleaning and disinfection

#### Cleaning

Clean the pad after each use. Clean using neutral warm soapy water. Ensure the pad is dry before storage. **DO NOT IMMERSE IN WATER.**



Clean using standard hospital wipes



Do not immerse in water

#### Disinfection

Disinfect using quaternary solution following the manufacturers recommendation for achieving low level disinfection. Wipe dry with a clean cloth.



Do not autoclave



**CAUTION:**  
Do not use cleaning solution at full strength



Do not apply tape to this pad



Do not use Bleach



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)



Medical Device



UK Conformity Assessed



Reuse



CE Mark



Latex Free



Non Sterile



5 060624 122057



Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Product Code: CSM-2576-20 Rev.6 02/24