



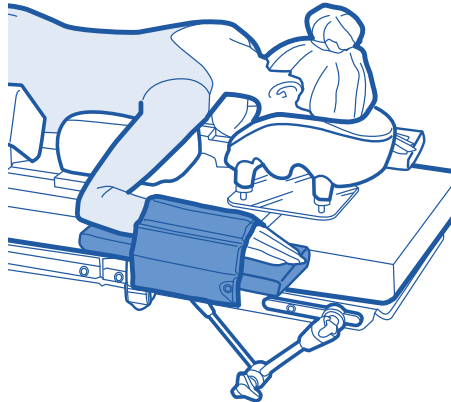

# CARE SURGICAL

## CS Universal Arm Support

Articulating armboard designed to enable safe and comfortable patient support in the prone, supine and lateral positions.

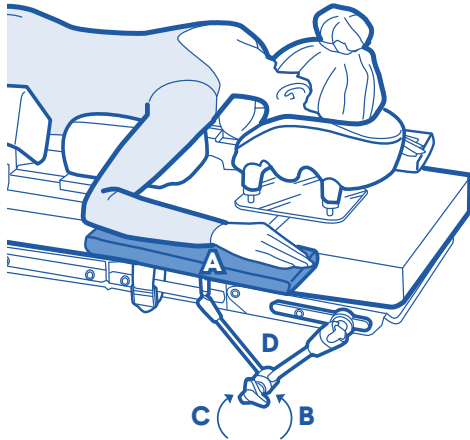

1. Hold the Universal Arm Support with one hand via the anterior segment (A) and manipulate the central clamping handle (D) with the other hand.
2. To release turn the central handle (D) counter clockwise (B) as far as necessary.
3. Move the Universal Arm Support into the desired position.
4. Turn the handle (D) clockwise (C) to secure into desired position.
5. Check that the Universal Arm Support is fixed firmly and is functioning correctly.
6. Use with compatible armboard pad for safe and comfortable positioning.




Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

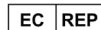

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



LOT Number:

Product Code: CSM-2646 Rev 5 02/24



Medical Device



Reuse



Latex Free



Non Sterile









UK Conformity Assessed



CE Mark

# CS Universal Arm Support - Positioning Solutions

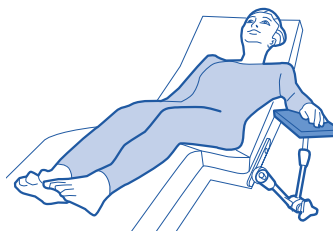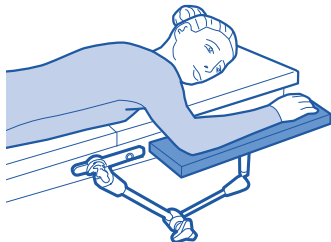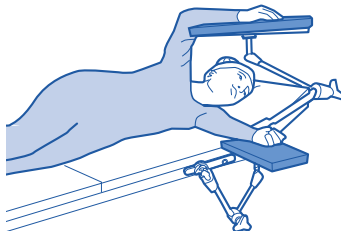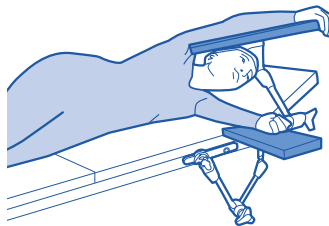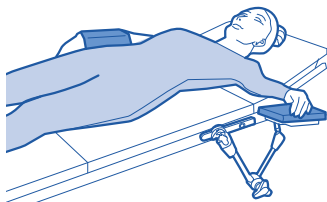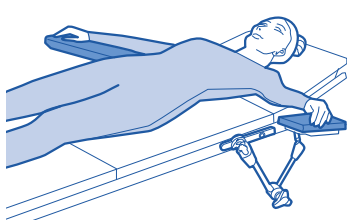


## Cleaning Instructions



Clean using standard  
hospital wipes



Do not immerse in water

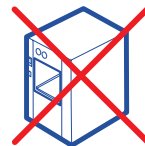

Do not autoclave



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
43 Broad Street  
Suite B106, Hudson,  
MA 01749, USA  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

EC REP

Care Surgical Supplies Limited, Office 2, 12a Lower Main  
Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity  
Assessed



CE Mark

Product Code: CSM-2646 Rev 5 02/24