





# CARE SURGICAL

# CS Head Strap Disposable

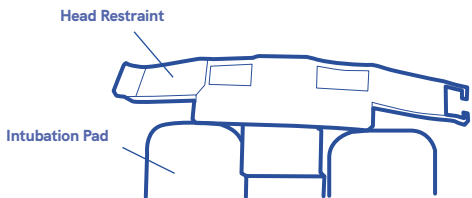


**1** Place CSM2667 (Head Strap) on intubation pad

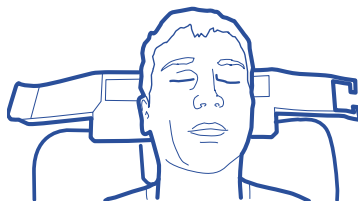


**2** Intubate patient in position




**3** Fold restraint over forehead

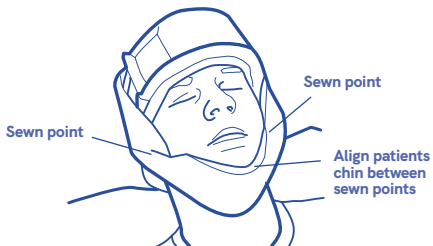

**4** Centre chin strap on patient

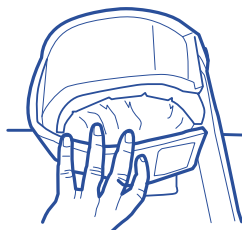

**5** Secure first strap over patient's head with Velcro®

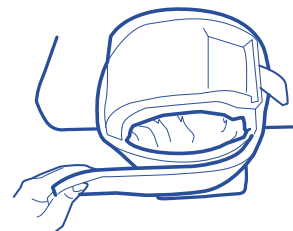

**6** Secure second strap over patient's head with Velcro®



Manufacturer

UK: Care Surgical Ltd  
Unit 6 Ringtail Road  
Burscough, L40 8JY  
Tel: 01704 336671

USA: Care Surgical LLC  
Marlborough  
MA 01752  
Tel: 866-243-4107

[www.care-surgical.com](http://www.care-surgical.com)

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,  
Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity Assessed



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120992

LOT:



Product Code: CSM-2667 Rev.4 09/23