CARE SURGICAL

Specialists in Patient Positioning

INSTRUCTIONS FOR USE

CSM2722 - 7

CS Supine Top (Base Only)

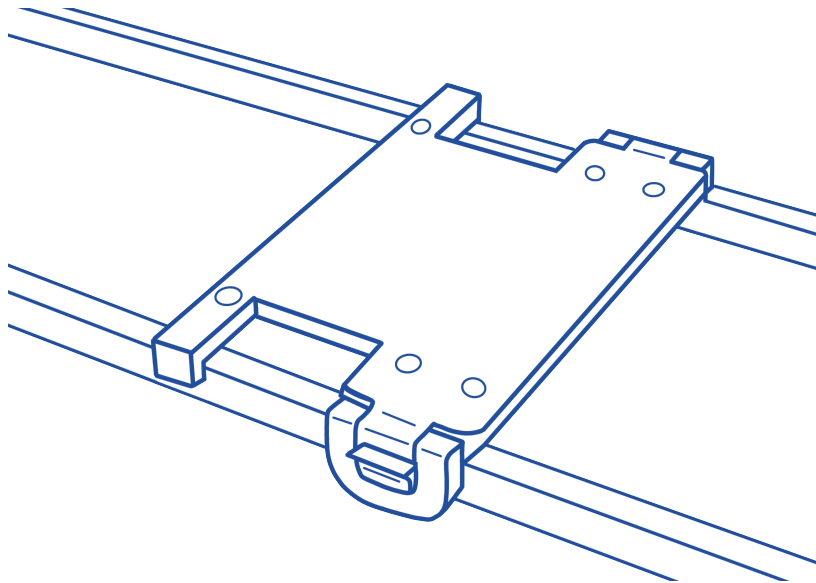
CARE SURGICAL

Product Code: CSM2722-7

CS Supine Top (Base Only)

 Only to be used by trained clinicians. Ensure product is secure before positioning patient.

The CS Supine Top (Base Only) can be attached to most dual-rail operating tables on the market*

For use with e.g. CS Supine Top Foam Pads CSM2722-2, and -3

*Compatible with rail sizes that measure 1.25" (3.2cm) wide and 1.5" (3.8 cm) tall with 14.5" (36.8 cm) space between. Allen® is the registered trademark of Allen Medical Systems Inc.

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.

DO NOT USE products that appear to be damaged or worn. In an event of product damage or patient injury, please contact Care Surgical.

5 060624 122392

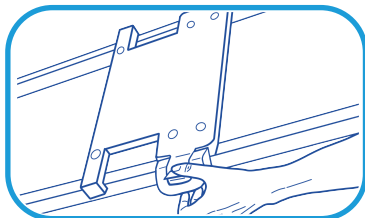
EU REP


CARE SURGICAL

Product Code: CSM2722-7


CS Supine Top (Base Only)

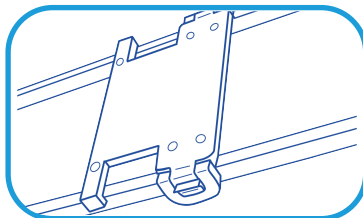- 1 Place the support on to the frame so that the brackets are over the bars.

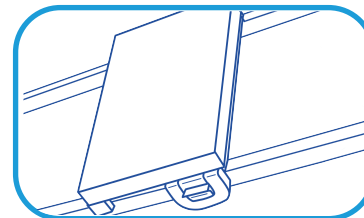- 2 Snap the locking clip shut to secure the support into position.

- 3 To adjust the position of the supports, release the clip(s), and slide the supports to the correct position, then snap lock the clip(s)

Cleaning instructions

Clean using standard hospital wipes

Do not immerse in water

Do not autoclave

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

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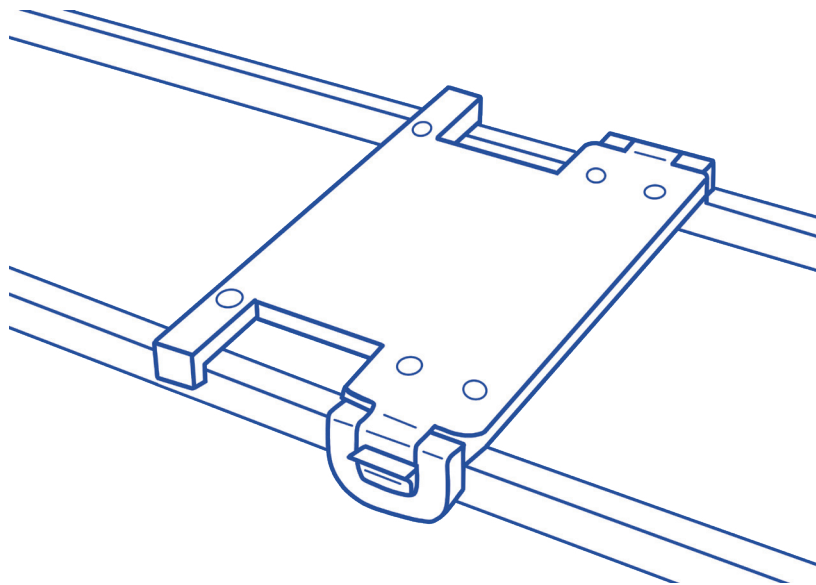
www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Irlanda

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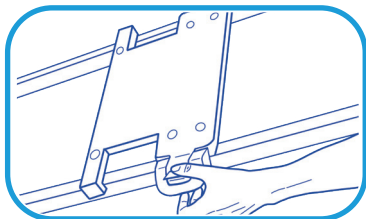
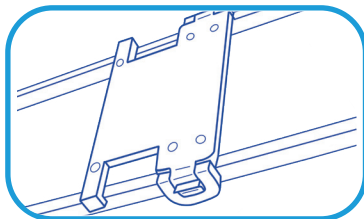
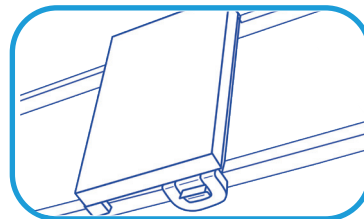
EU REP

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublino, K78 X5P8 Irlanda

CARE SURGICAL
Specialists in Patient Positioning

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Regno Unito: Care Surgical Ltd