





CARE SURGICAL

CS Head Strap Disposable

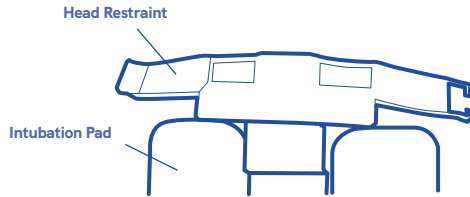


- 1 Place CSM2667 (Head Strap) on intubation pad

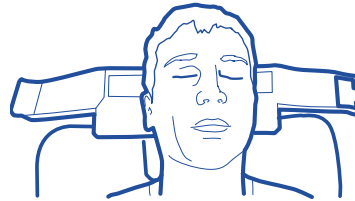

- 2 Intubate patient in position




- 3 Fold restraint over forehead

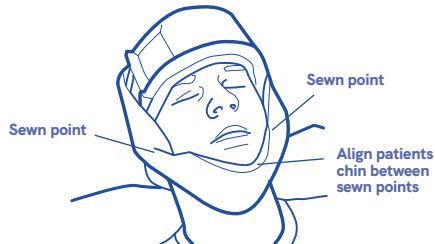

- 4 Centre chin strap on patient



- 5 Secure first strap over patient's head with Velcro®

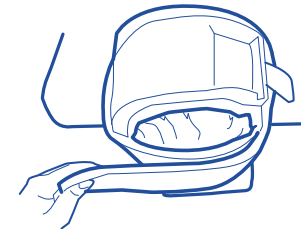

- 6 Secure second strap over patient's head with Velcro®



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity
Assessed



Do not reuse



CE Mark



Latex Free



Non Sterile



5 060624 120992



LOT:

EC REP

Product Code: CSM-2667 Rev. 5 01/24