




CARE SURGICAL

CS Supine Top (Base Only)

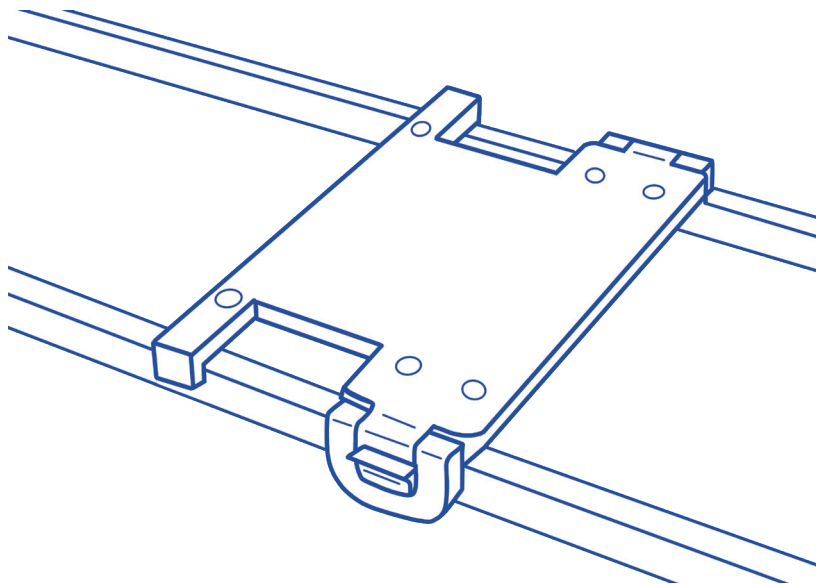

The CS Supine Top (Base Only) can be attached to most dual-rail operating tables on the market*

For use with e.g. CS Supine Top Foam Pads
CSM-2722-2, and -3

*Compatible with rail sizes that measure 1.25" (3.2cm) wide and 1.5" (3.8 cm) tall with 14.5" (36.8 cm) space between. Allen® is the registered trademark of Allen Medical Systems Inc.



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity
Assessed



Reuse



CE Mark



Latex Free



Non Sterile



5 060624 122392



LOT Number:


EC REP

Product Code: CSM-2722-7 Rev.2 01 /24




CARE SURGICAL


CS Supine Top (Base Only)

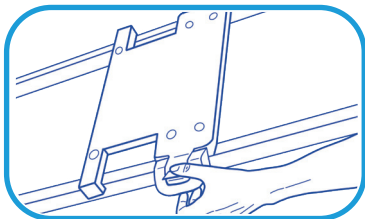

- 1** Place the attachment on the frame so that the brackets are over the bars.

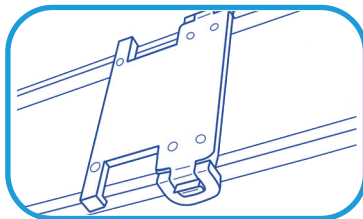

- 2** Snap the locking clip shut to secure the attachment in position.

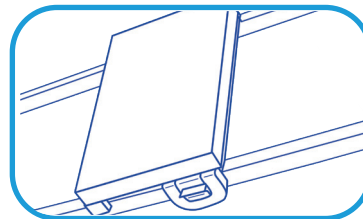

- 3** Place the CS Supine Top Foam Pad in the centre of the CS Supine Top Base.

Cleaning instructions



Clean using standard
hospital wipes



Do not immerse in water



Do not autoclave



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107



Medical Device



UK Conformity
Assessed



Reuse



CE Mark



Latex Free



Non Sterile



5 060624 122392

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

Product Code: CSM-2722-7 Rev.2 01 /24

EC REP