



CARE SURGICAL

CS Yellofin Compatible Replacement Pads



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower
Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

EC REP



Reuse



CE Mark



Latex Free



Non Sterile



5 060624 122132

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia

LOT Number:

Product Code: CSM-2734 Rev 2 04/22



Pad cleaning and disinfection

Cleaning

Clean the pad after each use. Clean using neutral warm soapy water. Ensure the pad is dry before storage. **DO NOT IMMERSE IN WATER**




Clean using standard hospital wipes



Do not immerse in water

Disinfection

Disinfect using quaternary solution following the manufacturers recommendation for achieving low level disinfection. Wipe dry with a clean cloth.

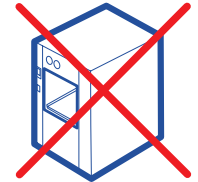

Do not autoclave



CAUTION:
Do not use cleaning solution at full strength



Do not apply tape to this pad



Do not use Bleach




Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
Marlborough
MA 01752
Tel: 866-243-4107

Care Surgical Supplies Limited,
Office 2, 12a Lower Main Street,
Lucan Co. Dublin, K78 X5P8 Ireland

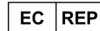

www.care-surgical.com

Australian Sponsor, Summit Surgical
PO Box 60, Landsborough
QLD 4550 Australia



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark



Maximum Weight