CARE SURGICAL
Specialists in Patient Positioning

INSTRUCTIONS FOR USE

CSM2734

CS Yellofin Compatible Replacement Pads


CARE SURGICAL

Product Code: CSM2734


CS Yellofin Compatible Replacement Pads

⚠ Only to be used by trained clinicians. Ensure product is secure before positioning patient.


CS Yellofin Compatible Replacement Pads ; use to support the patient's limb safely and comfortably whilst in the Yellofin Stirrup.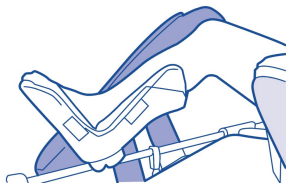

1 Position the patients leg and foot in the stirrup.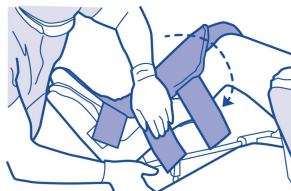

2 Ensuring the patient's leg is positioned correctly, take the stirrup pad straps and wrap the pad over and around the leg, enclosing the patients limb.

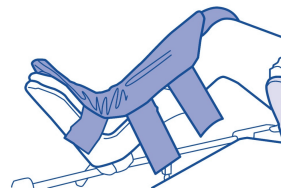3 Ensure the pad is attached securely utilising the Velcro® straps.

Cleaning instructions:

Clean using standard hospital wipes

Do not immerse in water

Do not autoclave

UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

EU REP

SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.

DO NOT USE products that appear to be damaged or worn.

In an event of product damage or patient injury, please contact Care Surgical.

5 060624 122132


CARE SURGICAL

Produktkode: CSM2734

CS Yellofin Compatible Replacement Pads

CS Yellofin Compatible Replacement Pads, Venstre (CSM-2964) og/eller Højre (CSM-2965); bruges til at støtte patientens ekstremitet sikkert og komfortabelt, mens den er i Yellofin Stirrup.

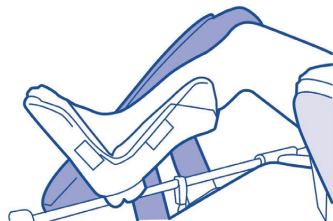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