



Technical Data Sheet

CSM2734 CS Yellofin Replacement Pads Pair



Description

- Reusable - clean using non-alcohol hospital wipes
- Secures and cushions patient's feet and legs in lithotomy stirrups
- Compatible with the Yellofin stirrups
- For use in a variety of surgical procedures
- Reduces the incidence of pressure ulcers and marking
- Maximum patient weight: 350lbs (159kg)

Application Area

- OR
- Anaesthesia

Measurements

- Length: 520mm/20.47"
- Width: 860mm/33.86"
- Height: 35mm/1.38"

Material Composition

- Polyurethane foam
- Latex & phthalate free
- Transfer coated PU with anti-bacterial treatment on knitted polyester

Cleaning

- Clean using non-alcohol hospital wipes

Order Information

Product Code	Product Name	Qty
CSM2734	CS Yellofin Replacment Pads Pair	1 Pair

Packaging Information

- Each pair is packaged in a recyclable polyethene bag
- 1 pair is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top
- Avoid storing where sharp objects could pierce the skin of the product

Contraindications

Incorrect positioning of the patient on this product can cause:

- Pressure marking or ulcers
- Neurological damage, which can cause paraplegia
- Venous pressure, which can lead to ischaemia

Examine the pad for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced.

- Do not autoclave
- Only to be used by trained medical personnel

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA:Y
- MD Class: 1
- UDI-DI/GTIN: 5060624122132

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson
MA 01749, USA
Tel: +1-866-243-4107



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity Assessed



CE Mark

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Australian Sponsor:

MDSS AU Pty Ltd, Ground Floor, Suite 55,
97-99 Bathurst Street, Sydney, NSW 2000, Australia