



CARE SURGICAL


CS Yellofin Compatible Replacement Pads

CS Yellofin Compatible Replacement Pads Left (CSM-2964) and/or Right (CSM-2965); use to support the patient's limb safely and comfortably whilst in the Yellofin Stirrup.




1

Position the patients leg and foot in the stirrup.

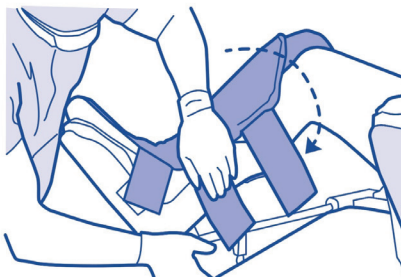

2

Ensuring the patient's leg is positioned correctly, take the stirrup pad straps and wrap the pad over and around the leg, enclosing the patients limb.

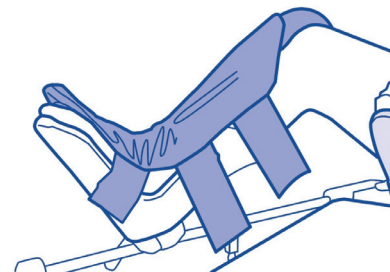

3

Ensure the pad is attached securely utilising the Velcro® straps.



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main
Street, Lucan Co. Dublin, K78 X5P8 Ireland



Medical Device



UK Conformity
Assessed



Reuse



CE Mark



Latex Free



Non Sterile



LOT Number:

Product Code: CSM-2734 Rev 2 02/24

EC REP



CARE SURGICAL

CS Yellofin Compatible Replacement Pads

Pad cleaning and disinfection

Cleaning

Clean the pad after each use. Clean using neutral warm soapy water. Ensure the pad is dry before storage. **DO NOT IMMERSE IN WATER**




Clean using standard hospital wipes



Do not immerse in water

Disinfection

Disinfect using quaternary solution following the manufacturers recommendation for achieving low level disinfection. Wipe dry with a clean cloth.

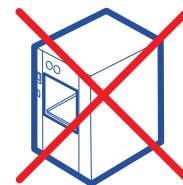

Do not autoclave



CAUTION:

Do not use cleaning solution at full strength



Do not apply tape to this pad



Do not use Bleach



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com



Medical Device



UK Conformity Assessed



Reuse



CE Mark



Latex Free



Non Sterile



5 060624 122132

EC REP

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Product Code: CSM-2734 Rev 2 02/24