




CARE SURGICAL


Product Code: CSM2739-1

CS Table Rail Adaptor-EU


⚠ Only to be used by trained clinicians. Ensure product is secure before positioning patient.

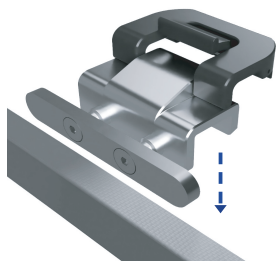

- 1** Position the rail adaptor in the desired position.

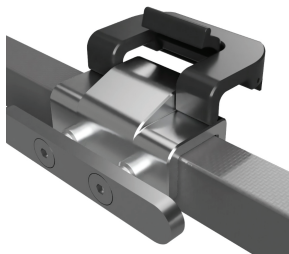


- 2** Place the rail adaptor flush to the operating table rail. Ensure the adaptor is placed with the clip on the inside facing the operating table and the adaptor facing outwards.

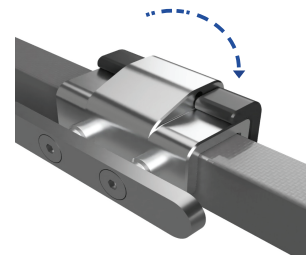

- 3** Close the secure locking clip to secure the adaptor ready for use.


Cleaning instructions:

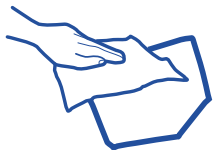

Clean using standard hospital wipes



Do not immerse in water

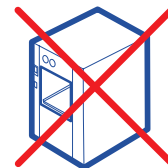

Do not autoclave



UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland



250kg
Maximum Patient Weight



SAFE DISPOSAL: customers should adhere to any local laws and regulations to the safe disposal of medical devices.
DO NOT USE products that appear to be damaged or worn.
In an event of product damage or patient injury, please contact Care Surgical.



5 060624 123573

Ref: CSM2739-1 - IFU (Rev.06) 03/2025