CARE SURGICAL
Specialists in Patient Positioning

INSTRUCTIONS FOR USE

CSM2647

CS Universal Arm Support - Moulded

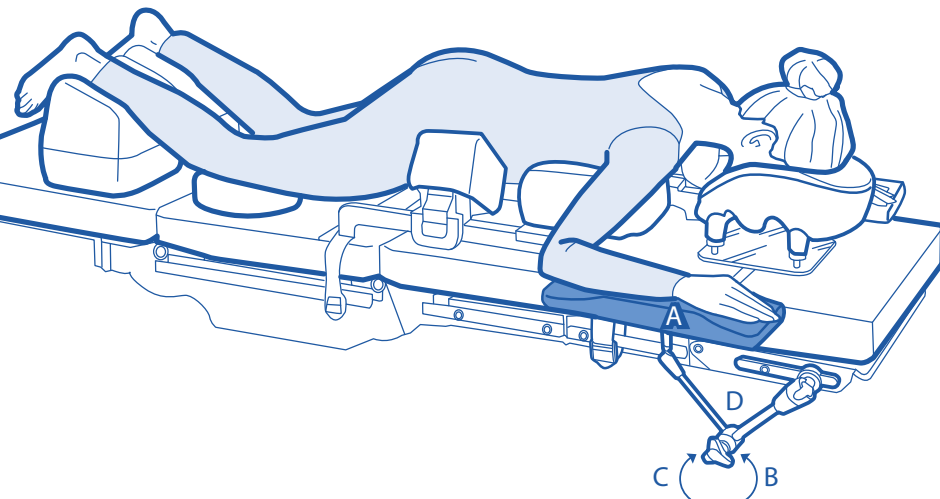
CARE SURGICAL

Product Code: CSM2647

CS Universal Arm Support™ - Moulded

⚠ Only to be used by trained clinicians. Ensure product is secure before positioning patient.

Articulating armboard designed to enable safe and comfortable patient support in the prone, supine and lateral positions.

1. Hold the Universal Arm Support™ with one hand via the anterior segment (A) and manipulate the central clamping handle (D) with the other hand.
2. To release turn the central handle (D) counter clockwise (B) as far as necessary.
3. Move the Universal Arm Support™ into the desired position.
4. Turn the handle (D) clockwise (C) to secure into desired position.
5. Check that the Universal Arm Support™ is fixed firmly and is functioning correctly.

UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

Maximum
Weight

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main
Street, Lucan Co. Dublin, K78 X5P8 Ireland

SAFE DISPOSAL: customers should adhere to any local laws
and regulations to the safe disposal of medical devices.

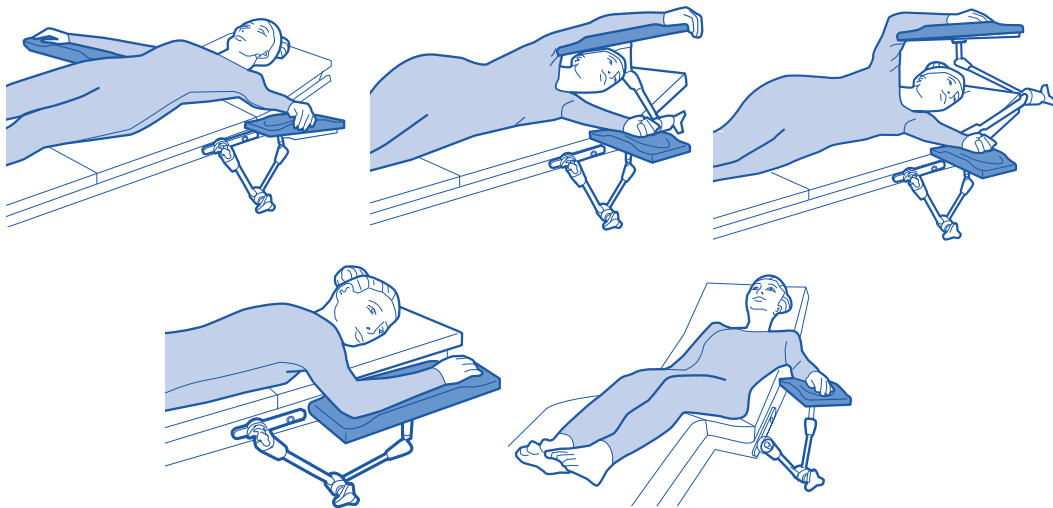
DO NOT USE products that appear to be damaged or worn.
In an event of product damage or patient injury,
please contact Care Surgical.

5 060624 123511

CS Universal Arm Support™ - Positioning Solutions.

Cleaning Instructions

Clean using standard
hospital wipes

Do not immerse in water

Do not autoclave

UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main
Street, Lucan Co. Dublin, K78 X5P8 Ireland

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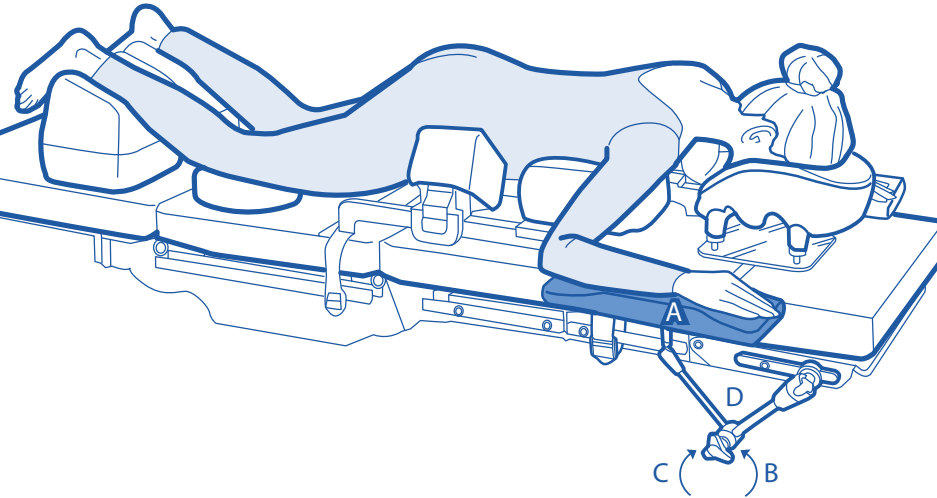
UK: Care Surgical Ltd
Unit 6 Ringtail Road,
Burscough,
L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

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