



Technical Data Sheet

CSM2647 CS Universal Arm Support - Moulded™



Description

- Supports the patient's arm when undergoing surgical procedures
- Reusable
- Attaches to most standard and specialist operating tables to support patients' arms in various positions during surgical procedures
- Number of uses: Unrestricted, subject to damage (see damage indicators in the Contraindications column)
- Supports maximum weight of 10kg
- Product weight 4.3kg

Application Area

- OR
- Anaesthesia

Measurements

- Length: 460mm/18.11"
- Width: 150mm/5.91"
- Height: 550mm/21.65"
- Max Height (Rail to Top) 510mm/20.07"

Material Composition

- Stainless Steel
- PU Foam
- ABS Plastic
- Aluminium
- PU Skin
- Latex & phthalate free

Order Information

Product Code	Product Name	Qty
CSM2647	CS Universal Arm Support - Moulded™	1 Unit

Packaging Information

- Each unit is packaged in a recyclable polyethene bag
- 1 unit is packaged into a cardboard box for sale

Storage Instructions

- First in first out
- Store in a cool dry place
- Avoid direct sunlight
- Do not store anything heavy on top
- Avoid storing where sharp objects could pierce the skin of the product

Contraindications

Incorrect positioning of the patient on this product can cause:

- Pressure marking or ulcers
- Neurological damage, which can cause paraplegia
- Venous pressure, which can lead to ischaemia
- Examine the pad for damage before and after each use. If the skin is pierced or torn, it cannot be cleaned effectively and should be replaced
- Do not immerse in water
- Do not autoclave
- Only to be used by trained medical personnel

Further Information

- Disposable/Reusable: Reusable
- Sterility: N
- CE/FDA/TGA/UKCA: Y
- MD Class: 1
- UDI-DI/GTIN: 5060624123511

Cleaning

- Clean using non-alcohol hospital wipes

Labelling



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
United Kingdom
Tel: +44 1704 336671

US: Care Surgical LLC
43 Broad Street
Suite B106, Hudson
MA 01749, USA
Tel: +1-866-243-4107

www.care-surgical.com



Care Surgical Supplies Ltd, Office 2, 12a Lower Main Street, Lucan Co. Dublin, K78 X5P8 Ireland

Australian Sponsor:
MDSS AU Pty Ltd, Ground Floor, Suite 55,
97-99 Bathurst Street, Sydney, NSW 2000, Australia



Medical Device



Reuse



Latex Free



Non Sterile



UK Conformity
Assessed



CE Mark