





CARE SURGICAL


CS Table Rail Adaptor-US

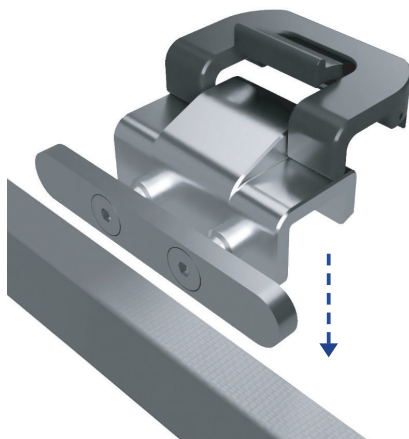

- 1** Position the rail adaptor in the desired position.

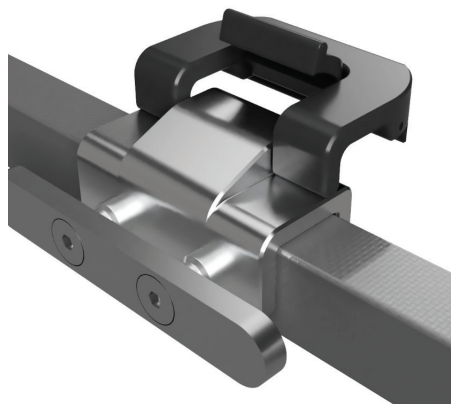

- 2** Place the rail adaptor flush to the operating table rail. Ensure the adaptor is placed with the clip on the inside facing the operating table and the adaptor facing outwards.

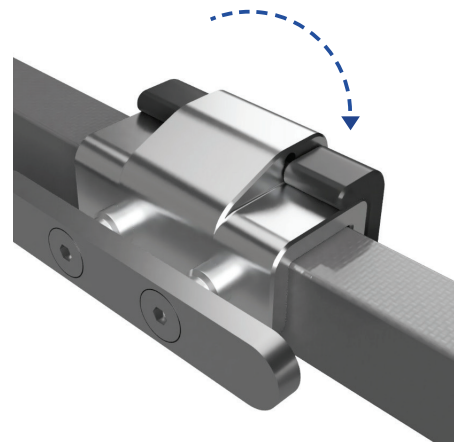

- 3** Close the secure locking clip to secure the adaptor ready for use.



Manufacturer

UK: Care Surgical Ltd
Unit 6 Ringtail Road
Burscough, L40 8JY
Tel: 01704 336671

USA: Care Surgical LLC
43 Broad Street
Suite B106, Hudson,
MA 01749, USA
Tel: 866-243-4107

www.care-surgical.com

Care Surgical Supplies Limited, Office 2, 12a Lower Main
Street, Lucan Co. Dublin, K78 X5P8 Ireland



Maximum
Weight



Medical Device



UK Conformity
Assessed



Reuse



CE Mark



Latex Free



Non Sterile



5 060624 123719



LOT Number:



Product Code: CSM-2739-3 Rev3 02/24